


Exploded views

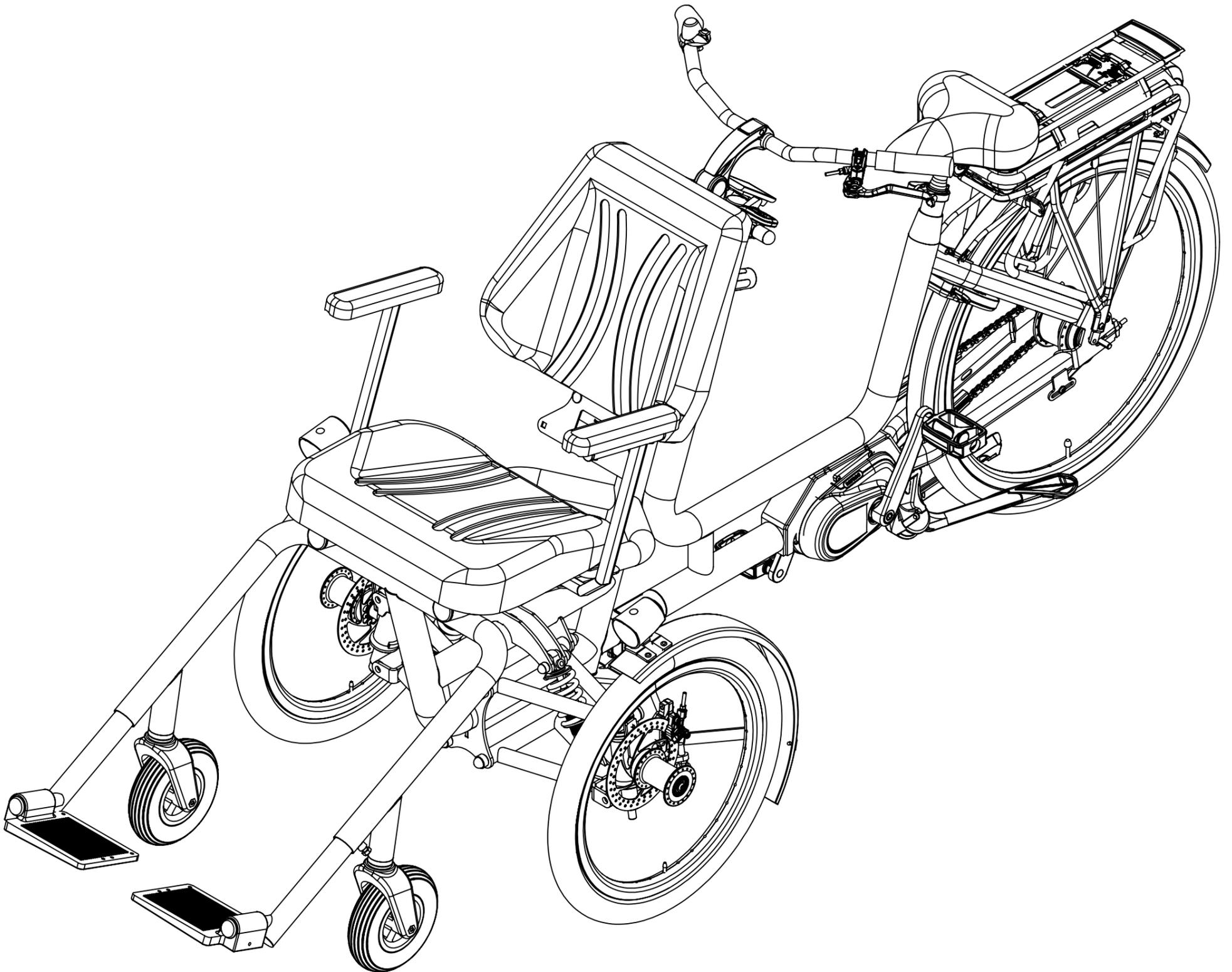

Data of issue: **16-5-2024**

Model: **Diaz**

Production period:
2022-present day

Be advised that in some cases the exploded views, might contain other parts beside what is shown in the illustration due to interchangeable accessories and options

This publication has been designed for use by Huka dealers, and may be updated periodically. Huka policy is one of continuous product improvement and the right is reserved to alter specifications at any time without prior notice. Whilst every care has been taken to ensure correctness of information, it is impossible to guarantee complete freedom from omissions and errors, or to accept liability arising from such errors and omissions




Huka BV

Munsterstraat 13
7575 ED Oldenzaal
The Netherlands

Huka BV, Münsterstraat 13, 7575 ED Oldenzaal, tel: +31 (0)541-572472, info@huka.nl




2	-	G01_14901-A200_00	
1	-	G01_14901-A100_00	
ITEM NO.	Artnr	DrawingNumber	Remark

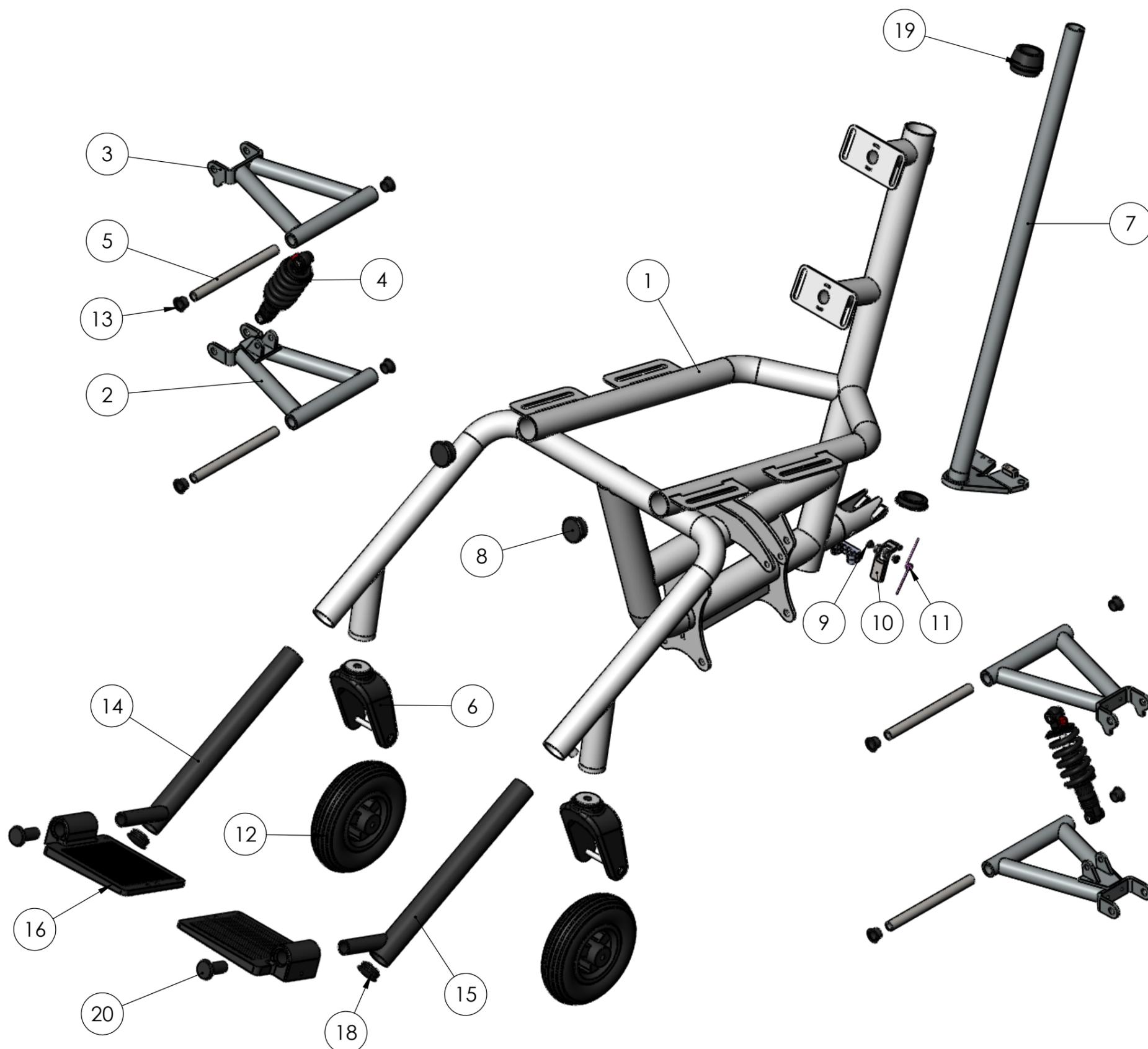


20	403023	Fixation plug for foot plate
19	403031	Headset bearing set 1 1/8 inch black
18	445041	Insert for tube Ø35mm black
17	403005	Footplate Diaz black
16	403005	Footplate Diaz black
15	403007	Leg rest tube Diaz left
14	403006	Leg rest tube Diaz right
13	403016	Sliding bearing 15x17x23mm
12	408451	Front wheel 200x50mm PU black
11	403015	Torsion spring 1,5x8,5x6,0mm
10	403400	Steering lock bracket Diaz
9	403029	Sliding bearing 6x8x12mm
8	401299	Insert for tube 40mm black
7	403053	Inner ball-head tube Diaz
6	408450	Front fork Diaz
5	403009	Long hinge tube Diaz
4	406256	Shock absorber 900lbs
3	403002	Wishbone frame top Diaz
2	403001	Wishbone frame bottom Diaz
1	403320	Wheelchair frame Diaz
POS	ART. NO.	NAME




24	-		G01_14901-A115_00
23	403058	Steering rod Diaz	
22	403021	Steering ball joint M10 Left SQ-10-L	
21	403020	Steering ball joint M10 Right SQ-10-R	
20	403049	Fusee arm bottom left	
19	403047	Fusee arm top left	
16	406265L	Hydraulic disc brake system left lever	
15	407120L	Adjustable ergonomic stem for handlebar	
11	426827L	Bicycle bell	
10	433012	Handlebar grip left	
9	433013	Handlebar grip right	
8	401424L	Handlebar large black	
3	-	G01_14901-A103_00	
2	-	G01_14901-A102_00	
1	-	G01_14901-A111_00	
POS	ART. NO.	NAME	REMARK OR DRAWING NUMBER

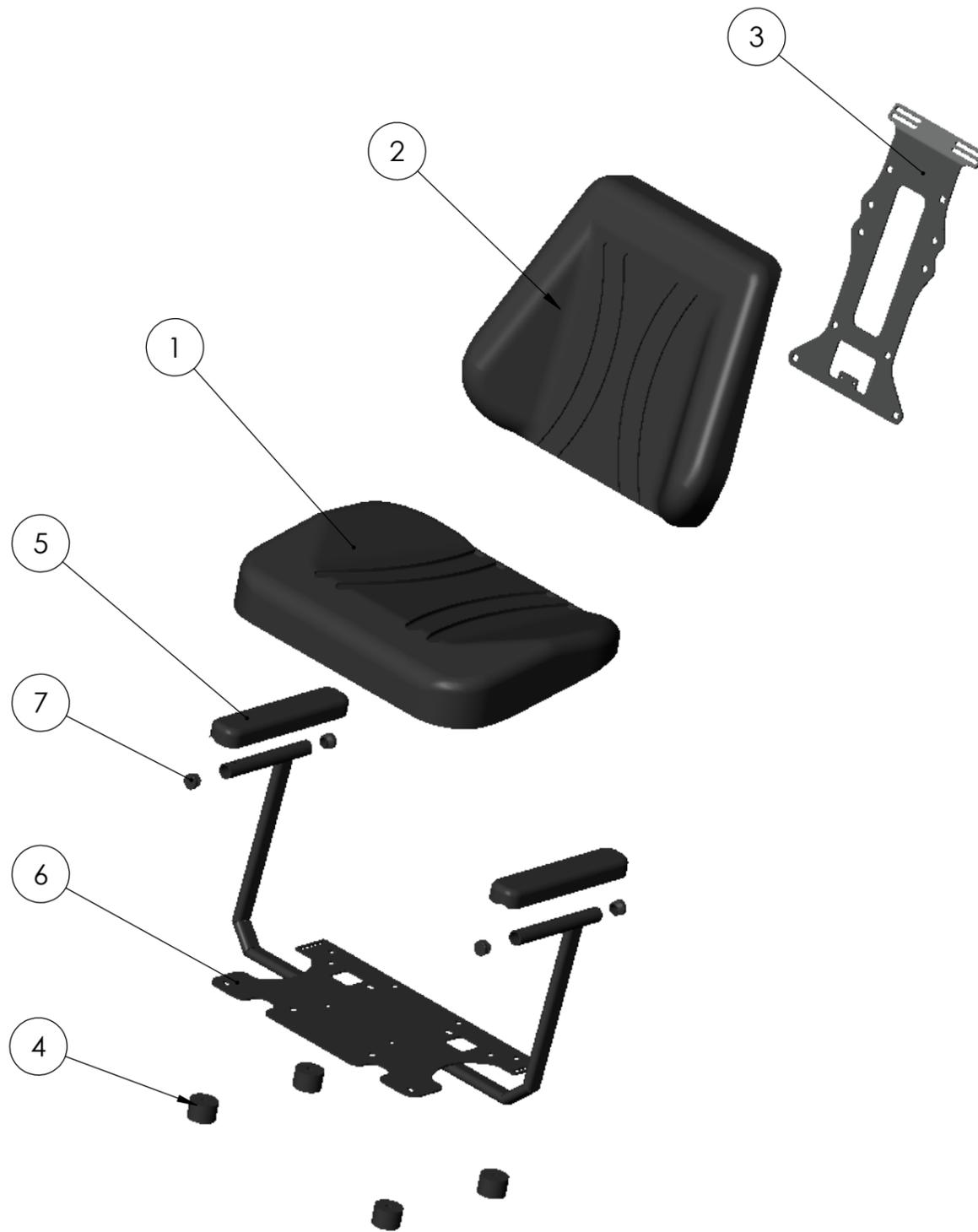


24	-	
7	477054	Insert for tube 22mm black
6	403018	Frame for not adjustable arm rest
5	404375	Armrest cushion PU foam
4	404621	Seat spacer nylon black
3	403019	Mounting plate for backrest
2	403003	Back cushion basic
1	403004	Seat cushion basic
POS	ART. NO.	NAME

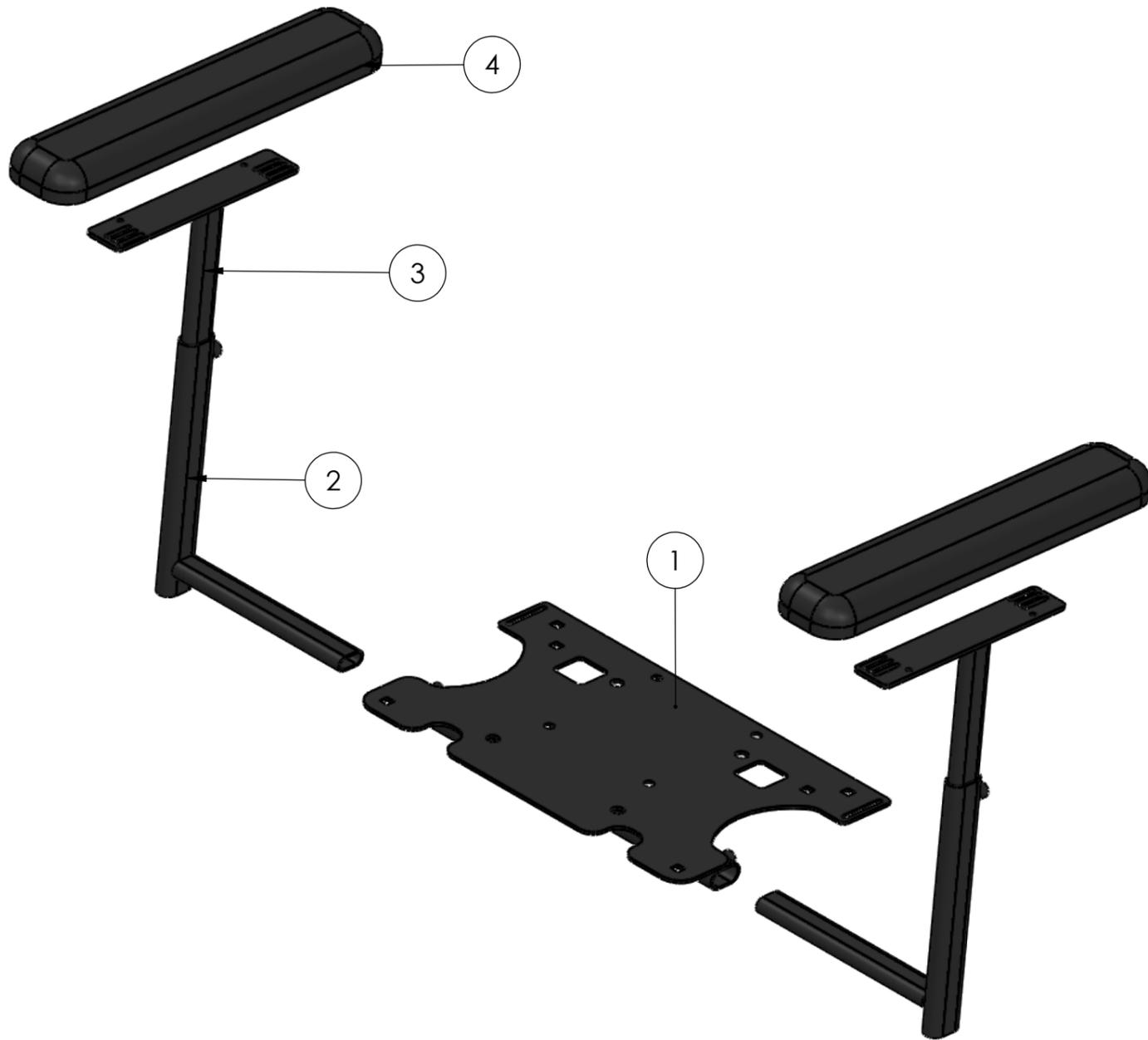


4	404376	Armrest cushion polstered 8x40cm pro skin
3	403037	Slider adapter set adjustable armrest
2	403036	Holder for armrest cushion
1	403035	Base plate for adjustable armrest
POS	ART. NO.	NAME

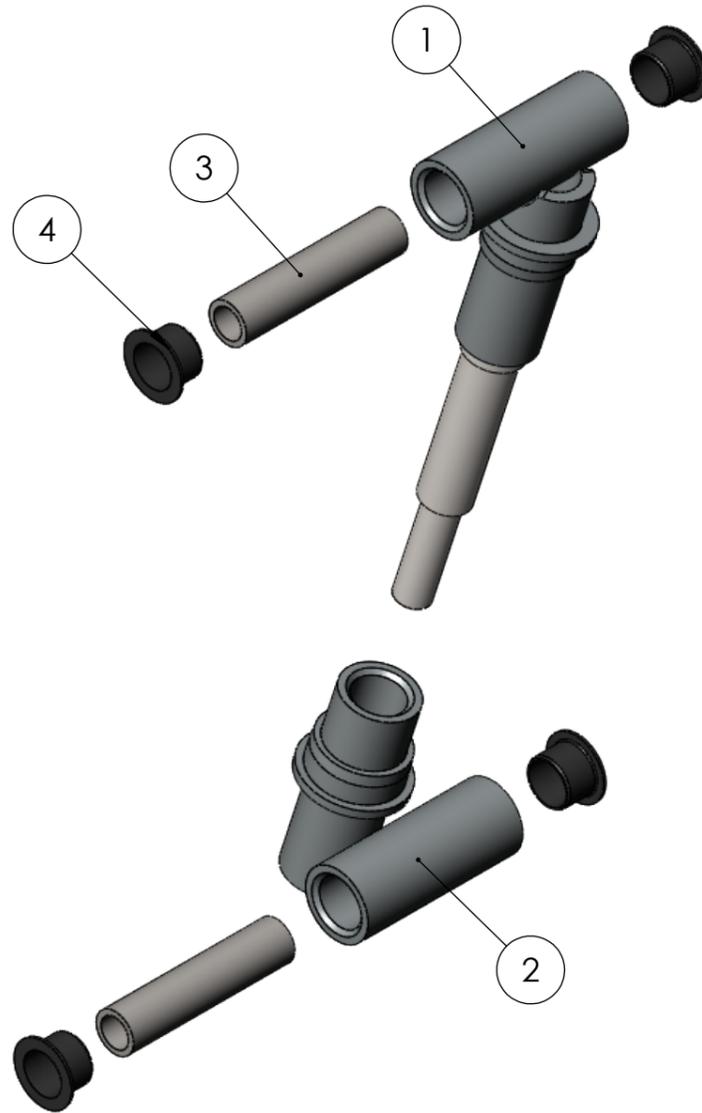


4	403016	Sliding bearing 15x17x23mm	
3	403009	Long hinge tube Diaz	G01_14901-P032_00
2	403050	Fusee arm bottom right	G01_14901-A114_00
1	403048	Fusee arm top right	G01_14901-A113_01
POS	ART. NO.	NAME	REMARK OR DRAWING NUMBER

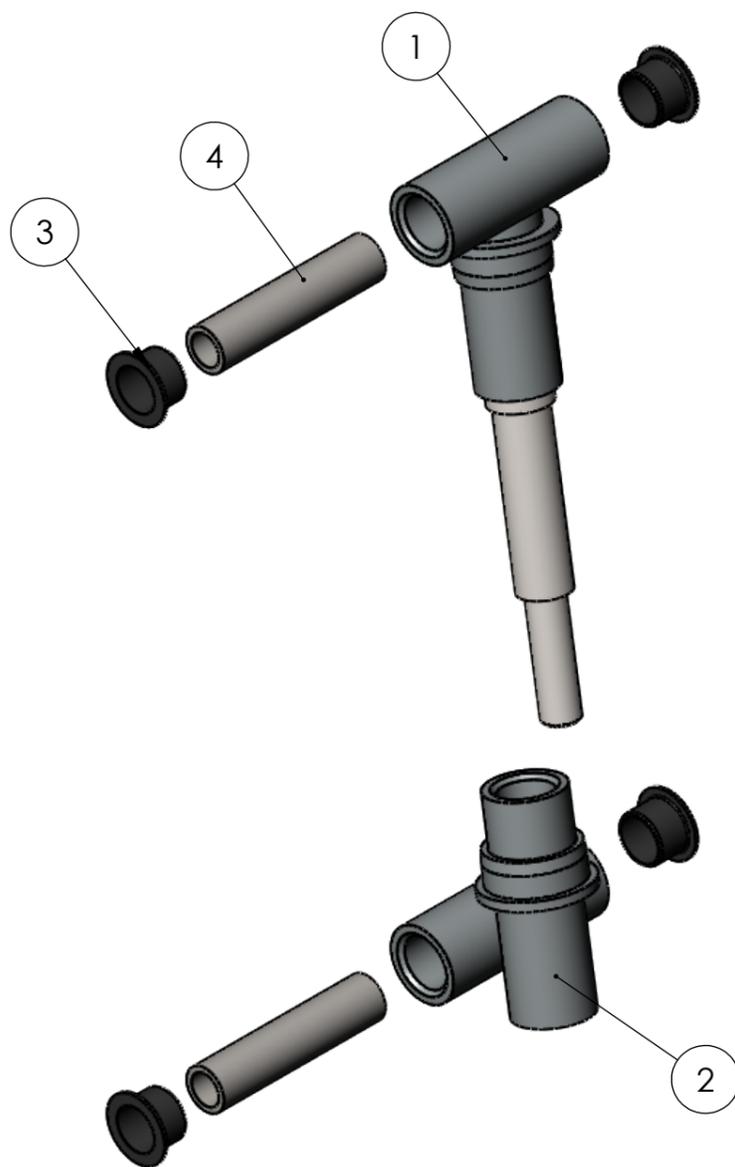


4	403009	Long hinge tube Diaz
3	403016	Sliding bearing 15x17x23mm
2	403049	Fusee arm bottom left
1	403047	Fusee arm top left
POS	ART. NO.	NAME

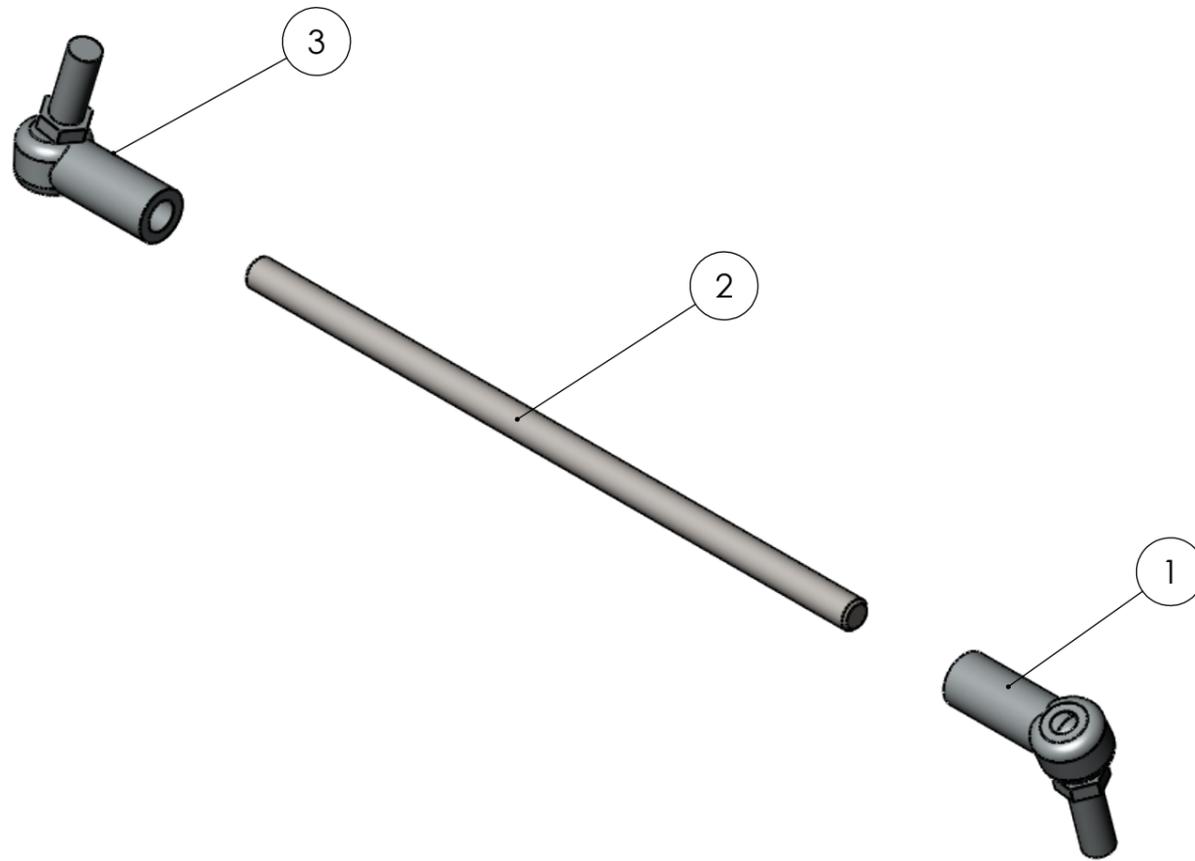


3	403020	Steering ball joint M10 Right SQ-10-R
2	403058	Steering rod Diaz
1	403021	Steering ball joint M10 Left SQ-10-L
POS	ART. NO.	NAME

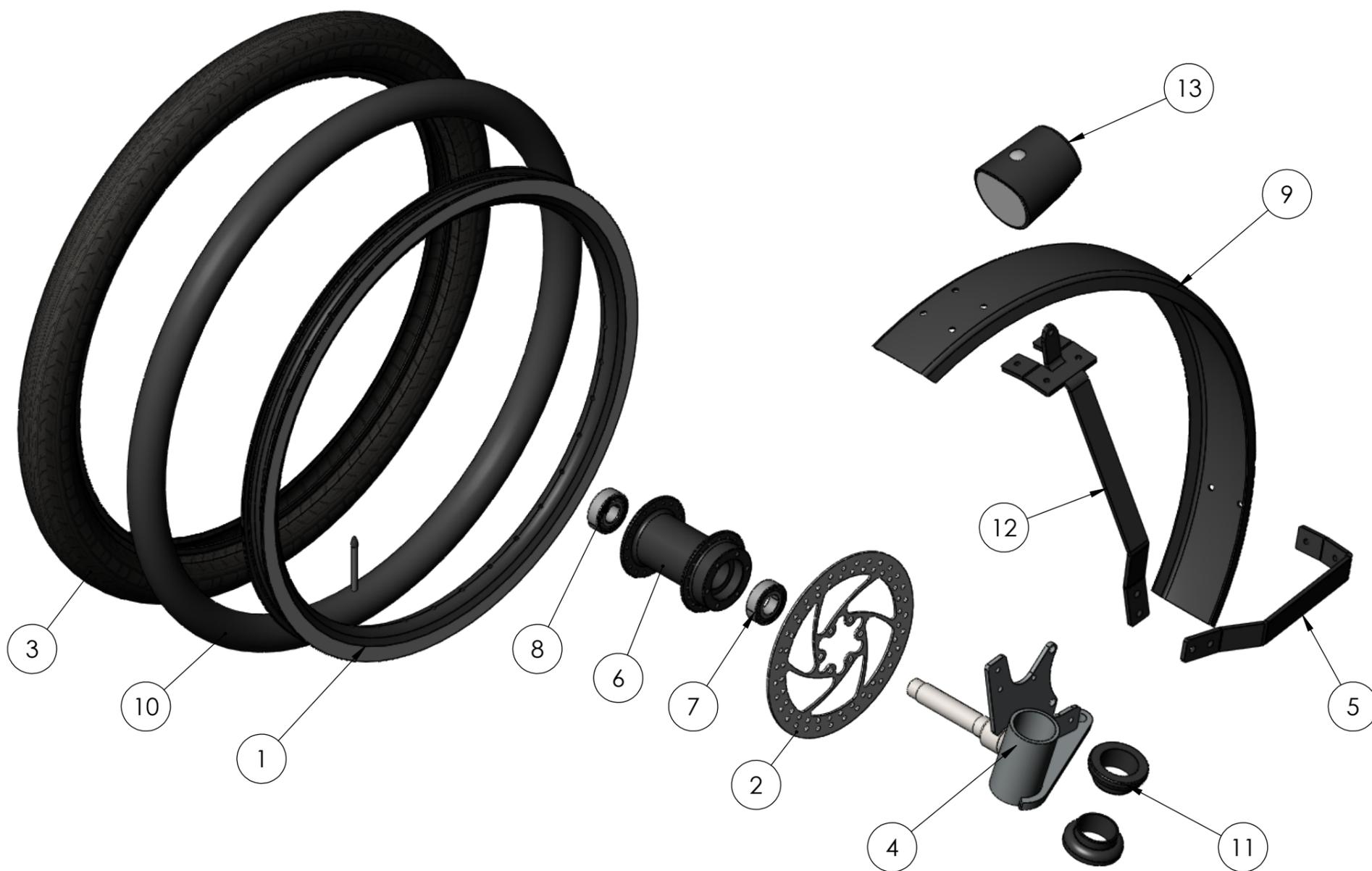


Left wheel has the same parts, except for Pos 4.

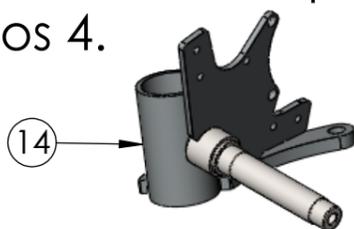

14	403051	Fusee left side Diaz	
13	432977	Headlight LED with battery	
12	403056	Fender holder incl. head light bracket	
11	403032	Ball-head bearing set A-head 1"	
10	401346	Inner tube 47-406 DV	
9	403011	Mudguard 20" front black Diaz	
8	431514	Ball-bearing 6202 2RS	
7	426278	Ball-bearing 6003 ZZ	
6	403022	Front wheel hub black	
5	403055	Fender holder Diaz	
4	403052	Fusee right side Diaz	
3	406303	Tire Big Ben plus 55-406	
2	406316	Brake disk 160mm Bengal	-
1	406227	Rim 20" (406) X-plorer	
POS	ART. NO.	NAME	REMARK OR DRAWING NUMBER

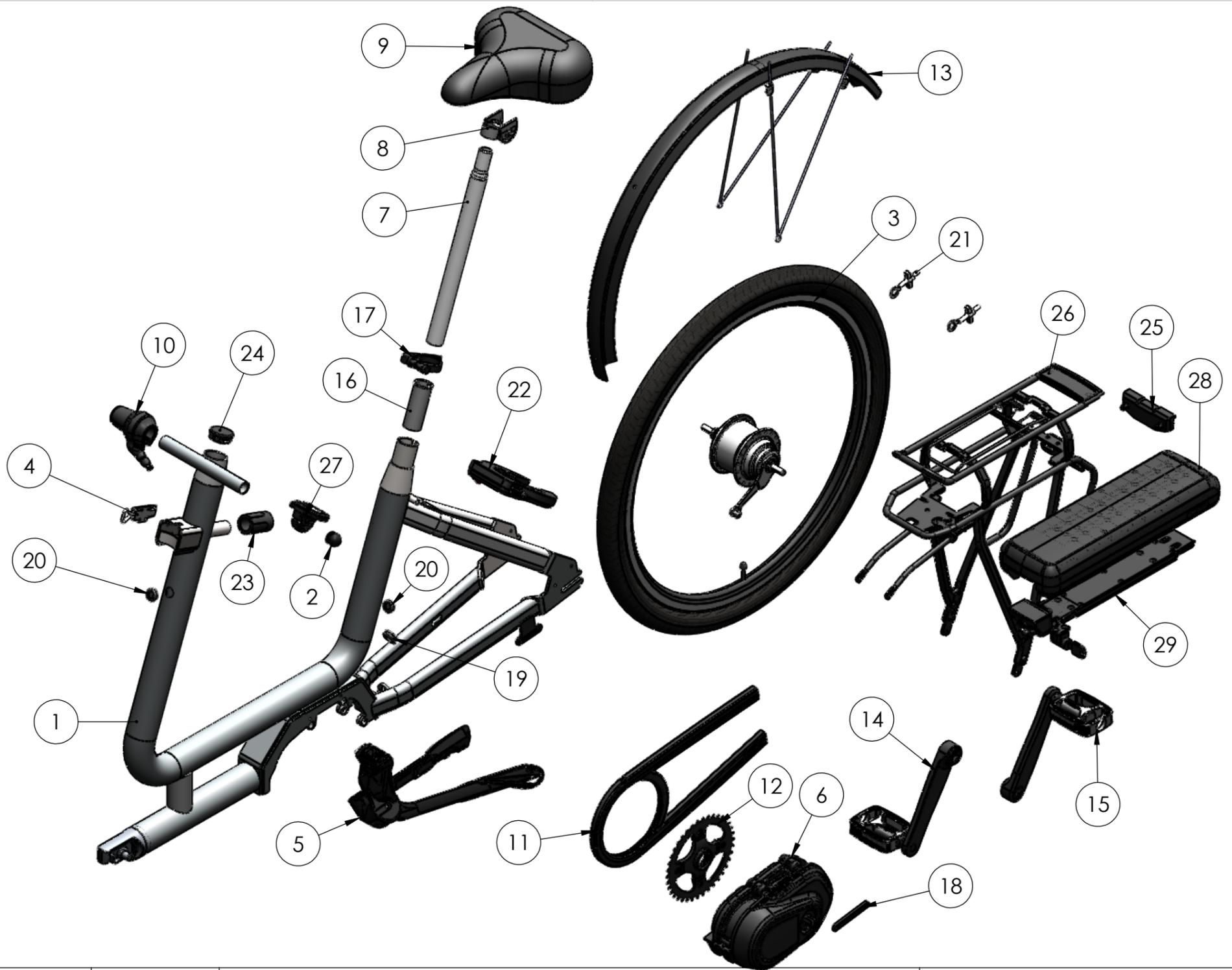

29	401934	Battery slider 232mm	401934
28	401932L	Battery Bafang 43V 750Wh	
27	401927L	Display Bafang C15	
26	401935	Rear carrier Bafang	
25	435144	Rear light for rear carrier e-bike	
24	401299	Plastic cover Ø40mm black	
23	412869	Rubber handgrip black	
22	426582	Safety lock axa solid	
21	480771	Chain tension adjuster	
20	403402	Tule 16mm	
19	403401	Tule 19mm	
18	401931	Mounting plate M400	146.81.05.0
17	401602	Clamp for seatpost	
16	451409	Filling socket 27,2 - 30mm	
15	401645L	Pedal set	
14	401937	Crank set Bafang 170mm	
13	401620	Mudguard 26" rear black	
12	401966	Chainwheel Bafang 38T 3/32	
11	401665	Chainglider	
10	403027	Gripshift nexus 5-speed	
9	401637L	Saddle velo plush	
8	411058	Saddle mounting set	
7	409033L	Seatpost Ø27,2 x 350mm	
6	401960	Mid motor Bafang M400 coaster brake	
5	490015	Two-leg stand	
4	403057	Clasp bracket Diaz	
3	403028	Rear wheel 26" Nexus 5-speed coaster brake	
2	410746	Plastic cover 22mm black	
1	403330	Frame Diaz black	G01_14901-A201_04
POS	ART. NO.	NAME	REMARK OR DRAWING NUMBER

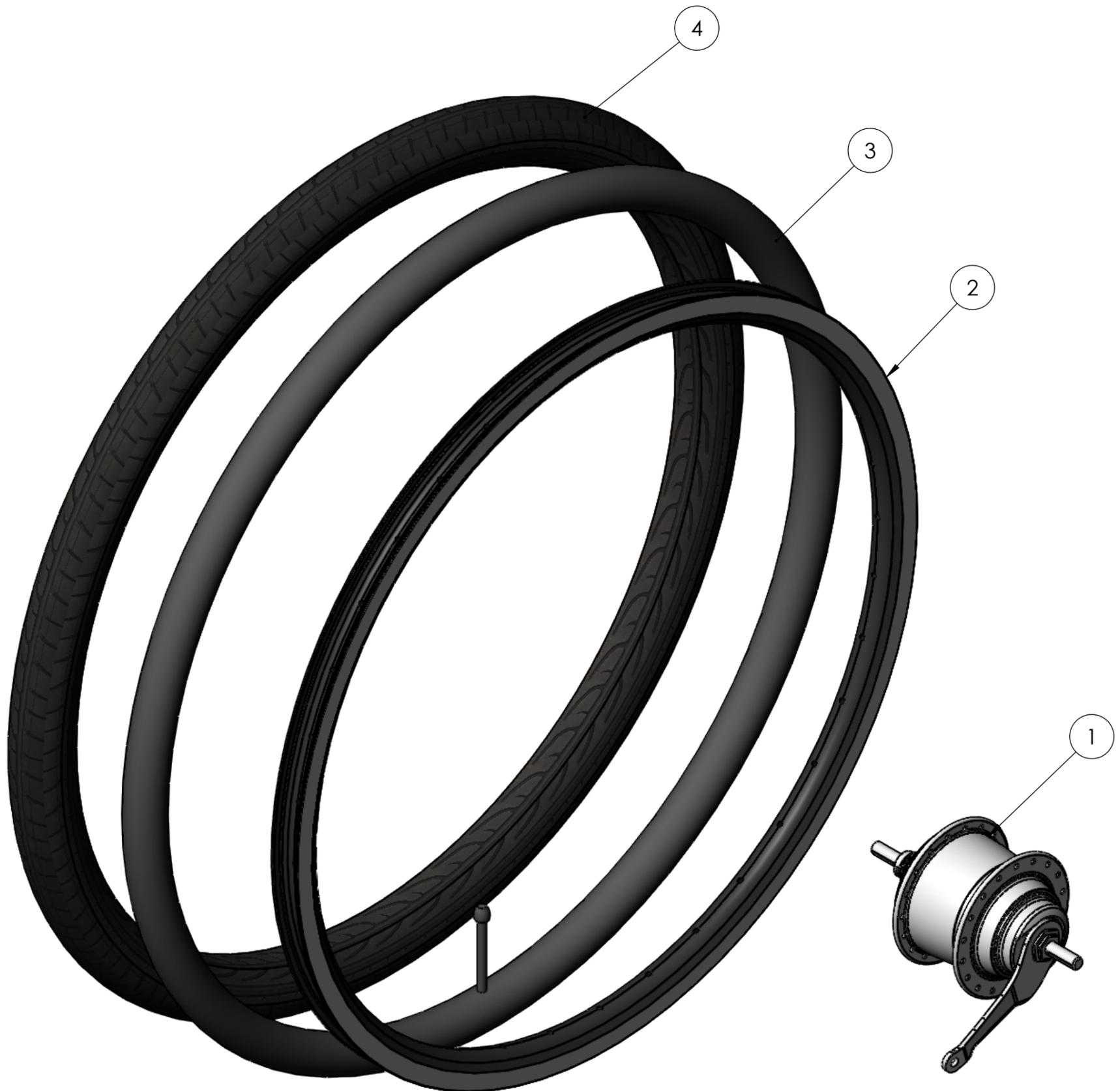

4	401338	Tire 26" Marathon Plus 47-559
3	401367	Inner tube 47-559 DV
2	426507	Rim 26" (559) X-plorer black
1	403026	Shimano 5-speed hub Nexus
POS	ART. NO.	NAME