



Exploded views

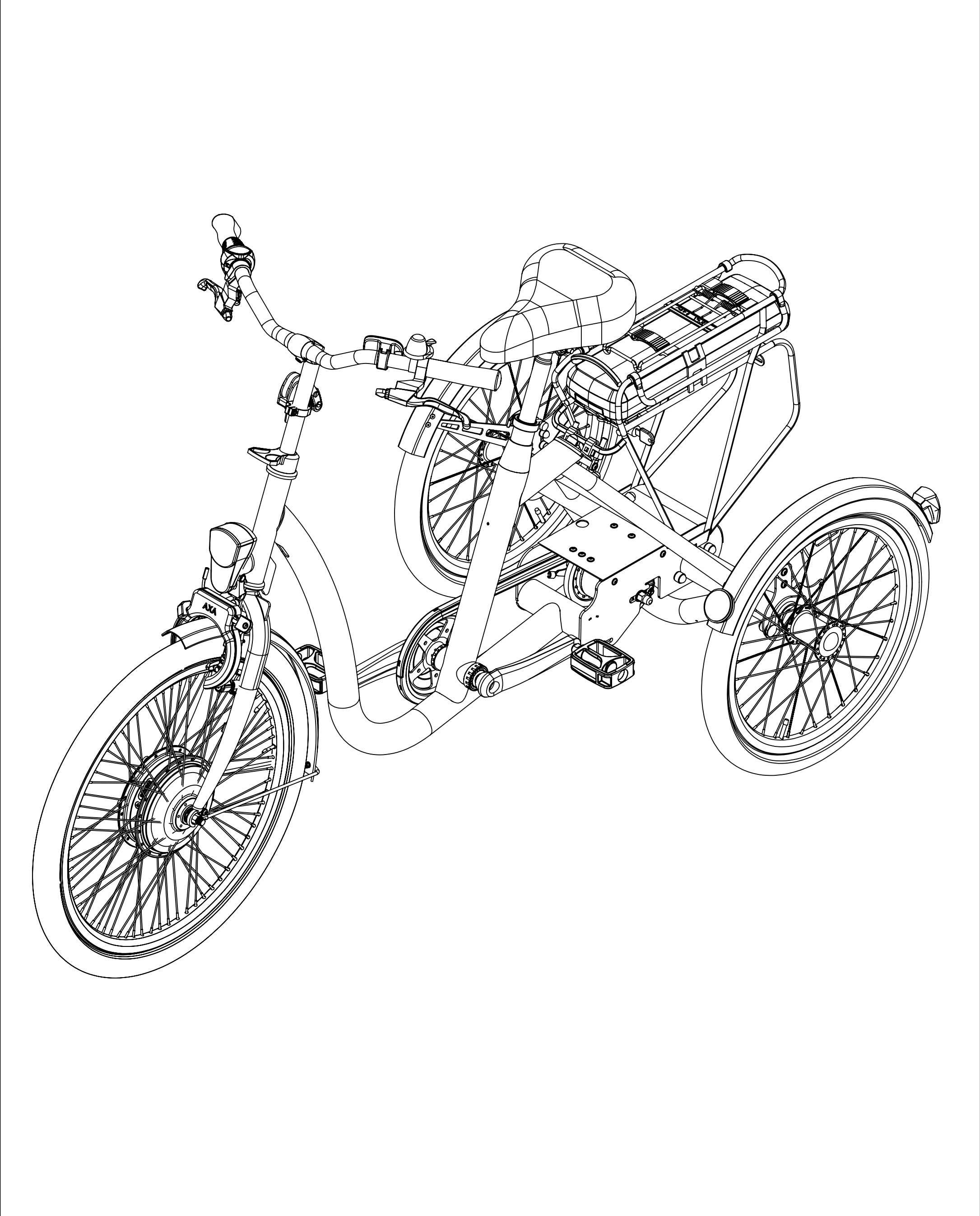


Data of issue: 25-6-2024	Be advised that in some cases the exploded views, might contain other parts beside what is shown in the illustration due to interchangeable accessories and options	 Huka BV Munsterstraat 13 7575 ED Oldenzaal The Netherlands
Model: C3	This publication has been designed for use by Huka dealers, and may be updated periodically. Huka policy is one of continuous product improvement and the right is reserved to alter specifications at any time without prior notice. Whilst every care has been taken to ensure correctness of information, it is impossible to guarantee complete freedom from omissions and errors, or to accept liability arising from such errors and omissions	
Production period: 2020-present day		
Huka BV, Münsterstraat 13, 7575 ED Oldenzaal, tel: +31 (0)541-572472, info@huka.nl		

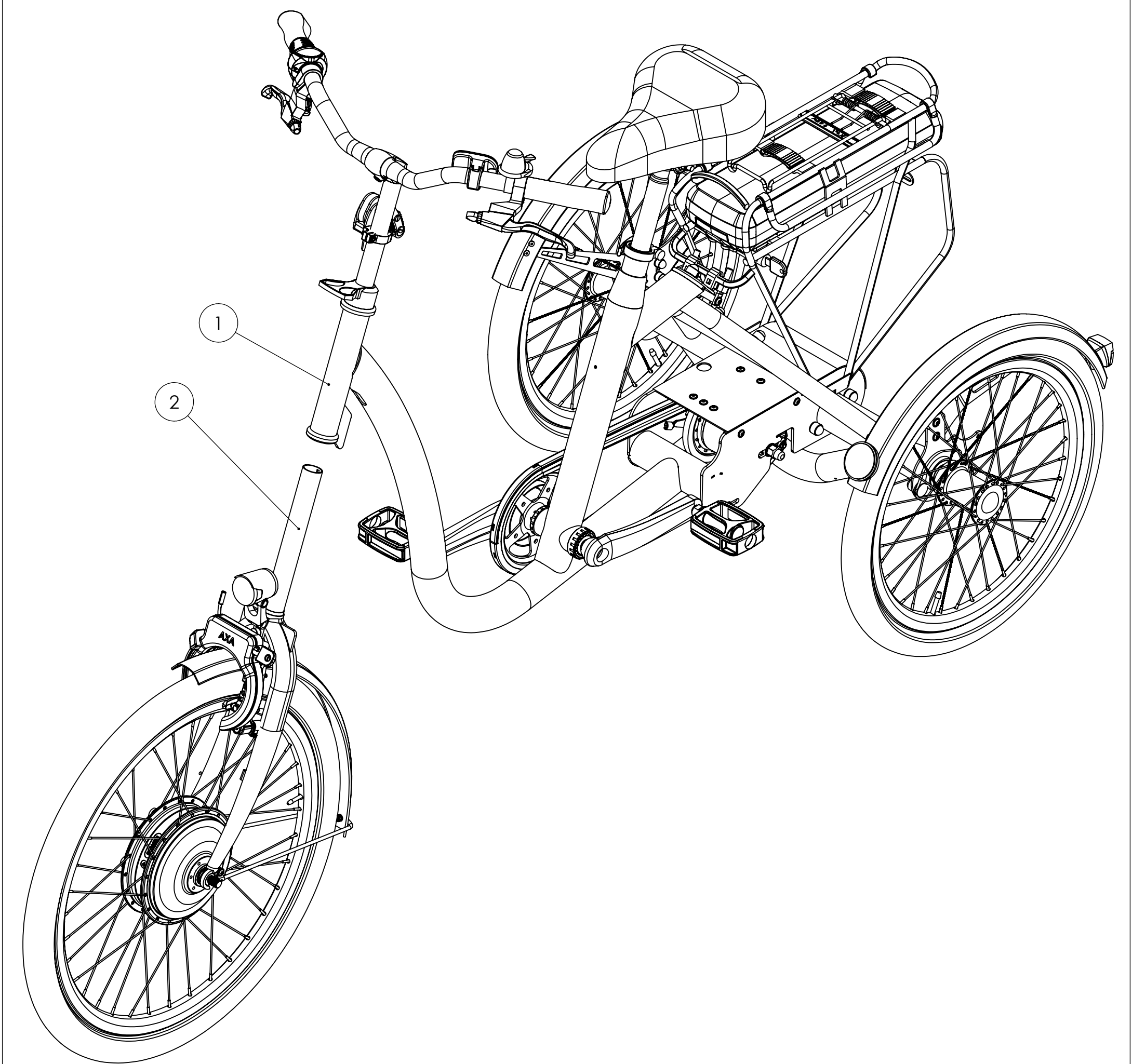

2	-	Front fork C3
1	-	Frame C3
POS	ART. NO.	NAME

Drawing number:


Drawing name: Front fork C3

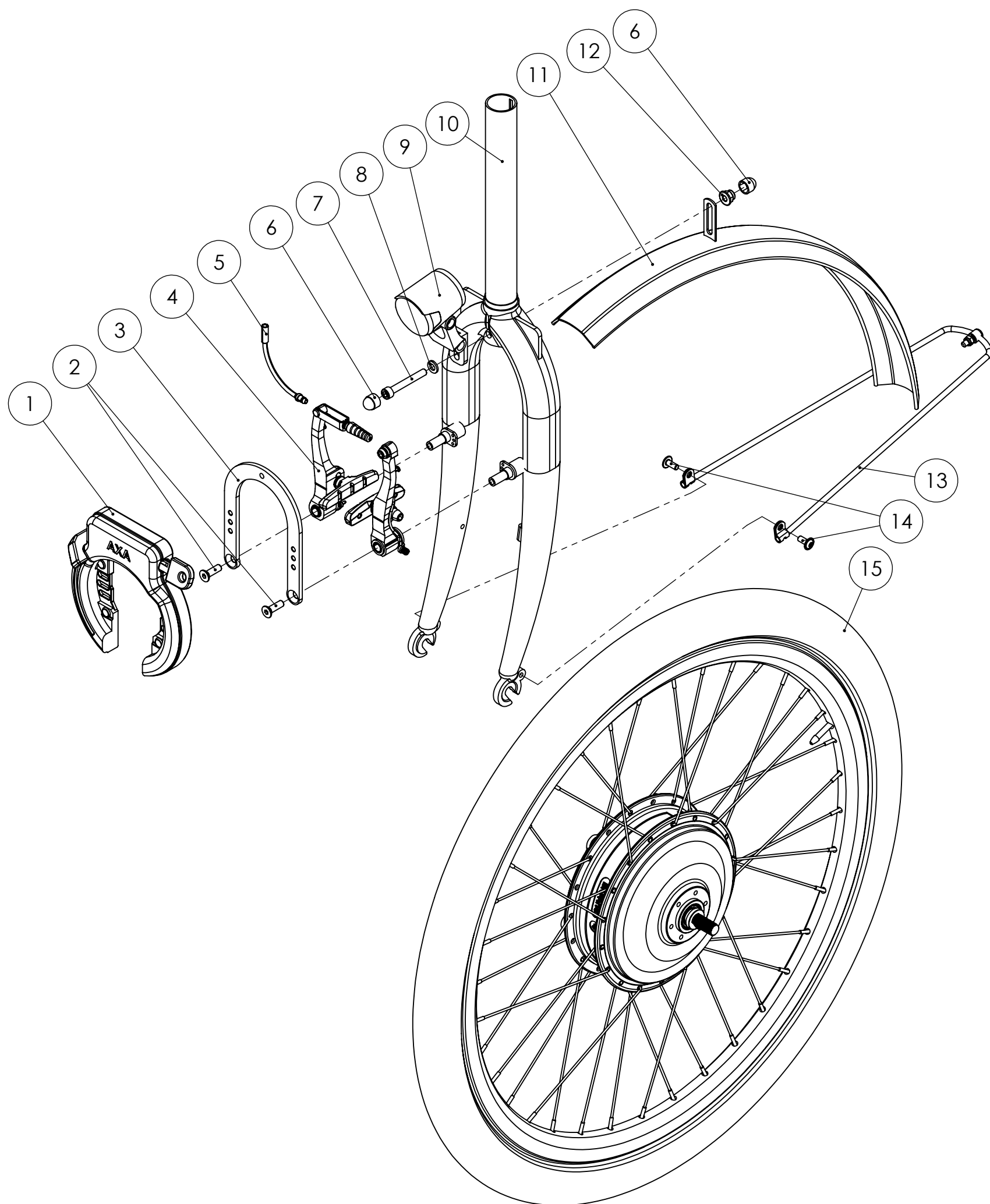


15	401950	Wheel 24" front
14	438268	Hex socket button head screw with flange ISO 7380-2 zinc plated M5 X 12 8.8
13	426763	Fender bracket
12	438267	Prevailing torque hex nut with flange DIN 6926 zinc plated M6 8.8
11	406231	Fender 24" front
10	404367	Front fork 24"
9	407210	Front light E-bike
8	410109	Plain washer DIN125-1A zinc plated M6
7	412963	Hex socket cap screw DIN 912 zinc plated M6 X 45 8.8
6	475222	Bolt protection cap M6
5	435063	V-brake noodle 90°
4	401528	Mechanical v-brake Shimano BR-T4000
3	404427	Ring lock bracket
2	464058	Hex socket countersunk head screw ISO 10642 zinc plated M6 X 20 8.8
1	426582	Ringlock AXA Solid
POS	ART. NO.	NAME

Huka BV, Münsterstraat 13, 7575 ED Oldenzaal, tel: +31 (0)541-572472, info@huka.nl


Published on: 25/06/24. Always consult www.huka.nl/wmo for the most up-to-date exploded views.

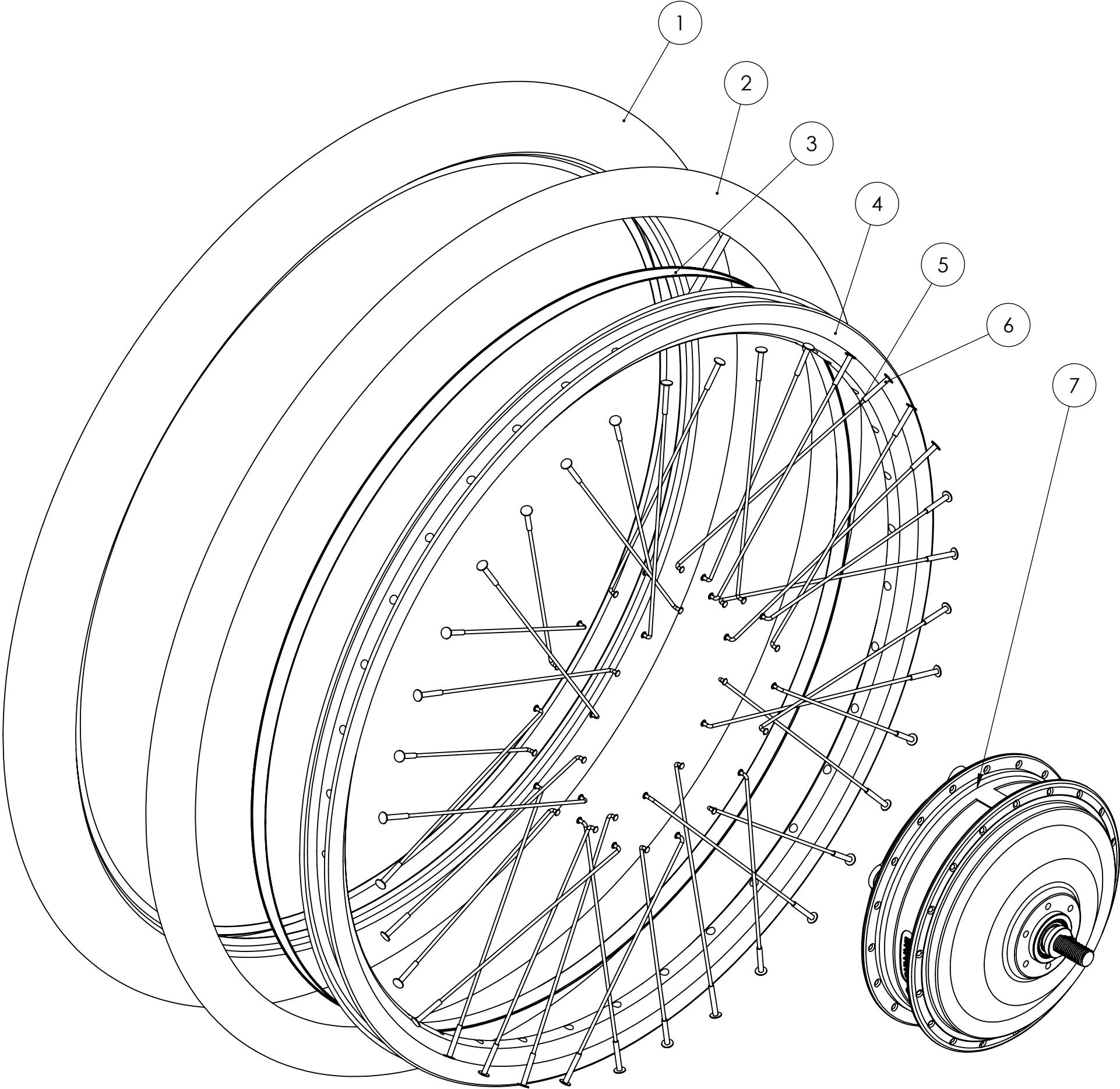


7	401920	Fronthub motor Bafang G070 250W
6	435067	Nipple 13G
5	401783	Spoke 13x163 stainless steel
4	426506	Rim 24" X-plorer
3	426530	Rim tape 15mm
2	401350	Inner tube 24" Schwalbe DV 55-507
1	401340	Tire 24" Schwalbe Marathon Plus 47-507
POS	ART. NO.	NAME

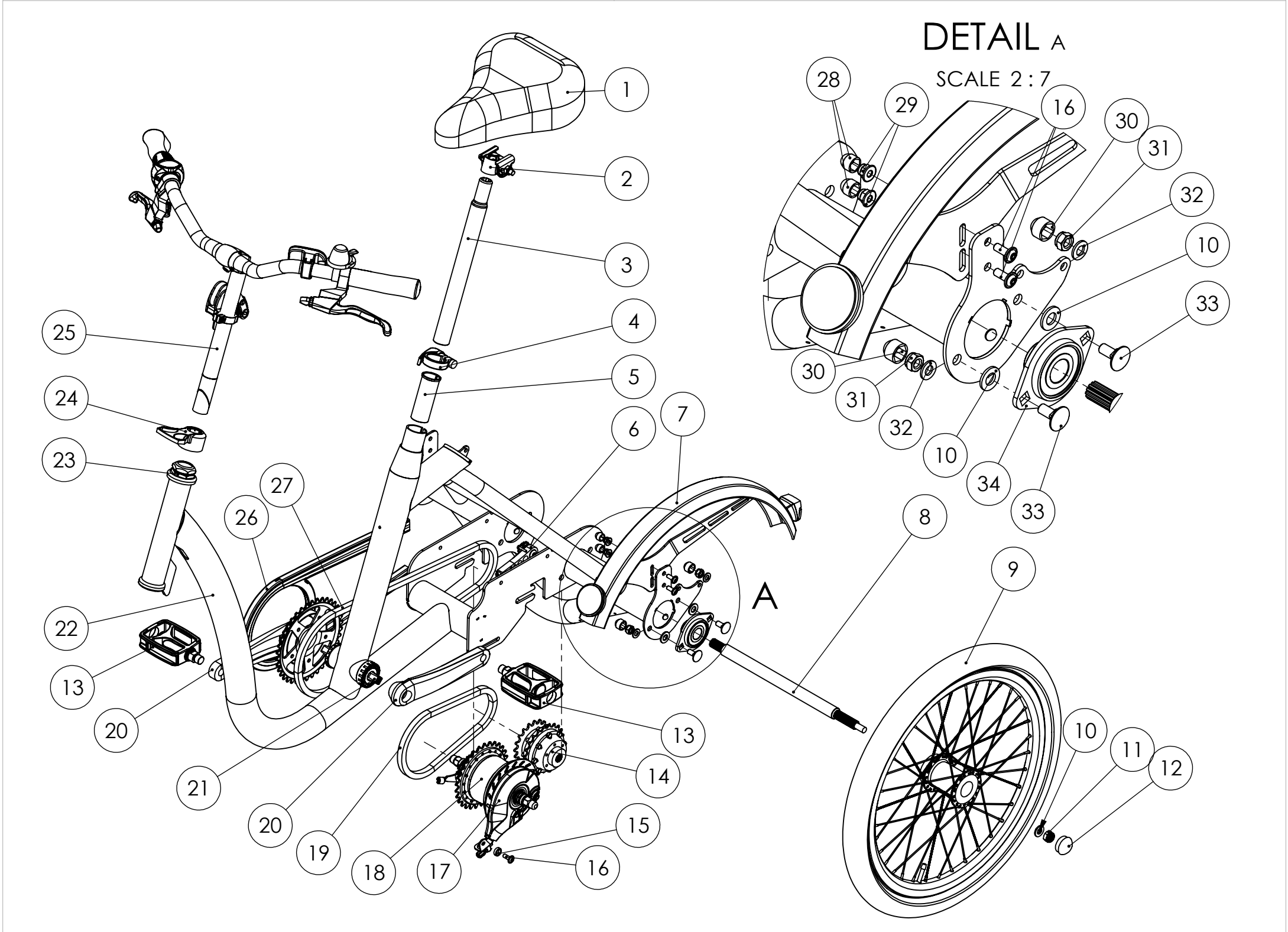


34	406217	Bearing flanged
33	410227	Carriage bolt DIN 603 zinc plated M8 X 20 4.8
32	410110	Plain washer DIN125-1A zinc plated M8
31	410204	Prevailing torque hex nut DIN 985 zinc plated M8
30	464065	Bolt protection cap M8
29	438267	Prevailing torque hex nut with flange DIN 6926 zinc plated M6 8.8
28	475222	Bolt protection cap M6
27	408058	Chain 1/2 x 3/32
26	401665	Chainguard chain glider
25	-	Steer
24	401666	Cable guide head tube 1 1/8"
23	409302	Headset bearing threaded 1 1/8 inch
22	401652G	Welded frame
21	401933	Speed sensor Bafang
20	408024	Crankset 38T 170mm
19	426808	Chain 1/2 x 1/8
18	406120VM	Gear hub Nexus 8V freewheel set
17	435005	Rollerbrake Shimano
16	451345	Hex socket button head screw with flange ISO 7380-2 zinc plated M6 X 16 8.8
15	497023	Shim ring 14 X 10 X 4
14	406405VM	Differential with spline
13	401645	Pedal set standard
12	411300	Protection cap Ø30
11	412970	Prevailing torque hex nut DIN 985 zinc plated M10
10	410080	Plain washer DIN125-1A zinc plated M10
9	401662	Wheel 20" Rear
8	406410	Rear axle left
7	406330	Fender left back
6	406249	Chain tensioner
5	451409	Filling socket 27,2 - 30mm
4	401602	Clamp for seatpost
3	409033	Seatpost rigid Ø27.2x350
2	411058	Saddle mounting set
1	401638	Saddle SR Sky
POS	ART. NO.	NAME

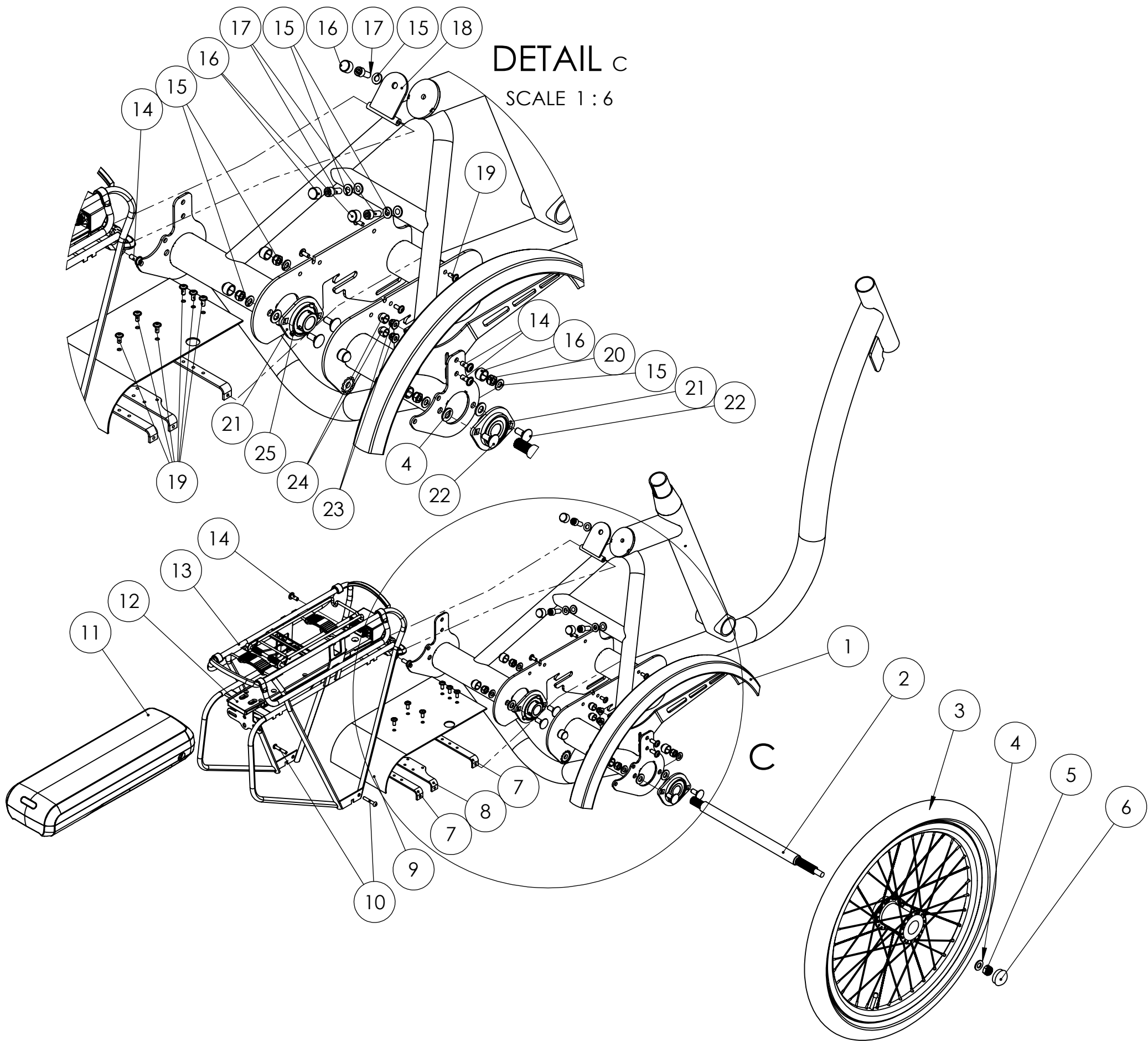


25	412965	Hex socket set screw DIN 913 zinc plated M6 X 6
24	475222	Bolt protection cap M6
23	438267	Prevailing torque hex nut with flange DIN 6926 zinc plated M6 8.8
22	410227	Carriage bolt DIN 603 zinc plated M8 X 20 4.8
21	406217	Bearing flanged
20	410204	Prevailing torque hex nut DIN 985 zinc plated M8
19	438268	Hex socket button head screw with flange ISO 7380-2 zinc plated M5 X 12 8.8
18	401659	Bracket carrier rack Bafang
17	410085	Hex socket cap screw DIN 912 zinc plated M8 X 20 8.8
16	464065	Bolt protection cap M8
15	410110	Plain washer DIN125-1A zinc plated M8
14	451345	Hex socket button head screw with flange ISO 7380-2 zinc plated M6 X 16 8.8
13	401935	Pannier rack Bafang
12	401934	Battery slider Bafang
11	401930	Battery Bafang 43V 450Wh
10	451344	Hex socket button head screw ISO 7380-1 stainless steel M5 x 30 A2
9	401656G	Chain guard back
8	401658	Support strip brake splitter
7	401655	Support strip chainguard
6	411300	Protection cap Ø30
5	412970	Prevailing torque hex nut DIN 985 zinc plated M10
4	410080	Plain washer DIN125-1A zinc plated M10
3	401663	Wheel 20" Rear
2	406411	Rear axle right
1	406331	Fender right back
POS	ART. NO.	NAME

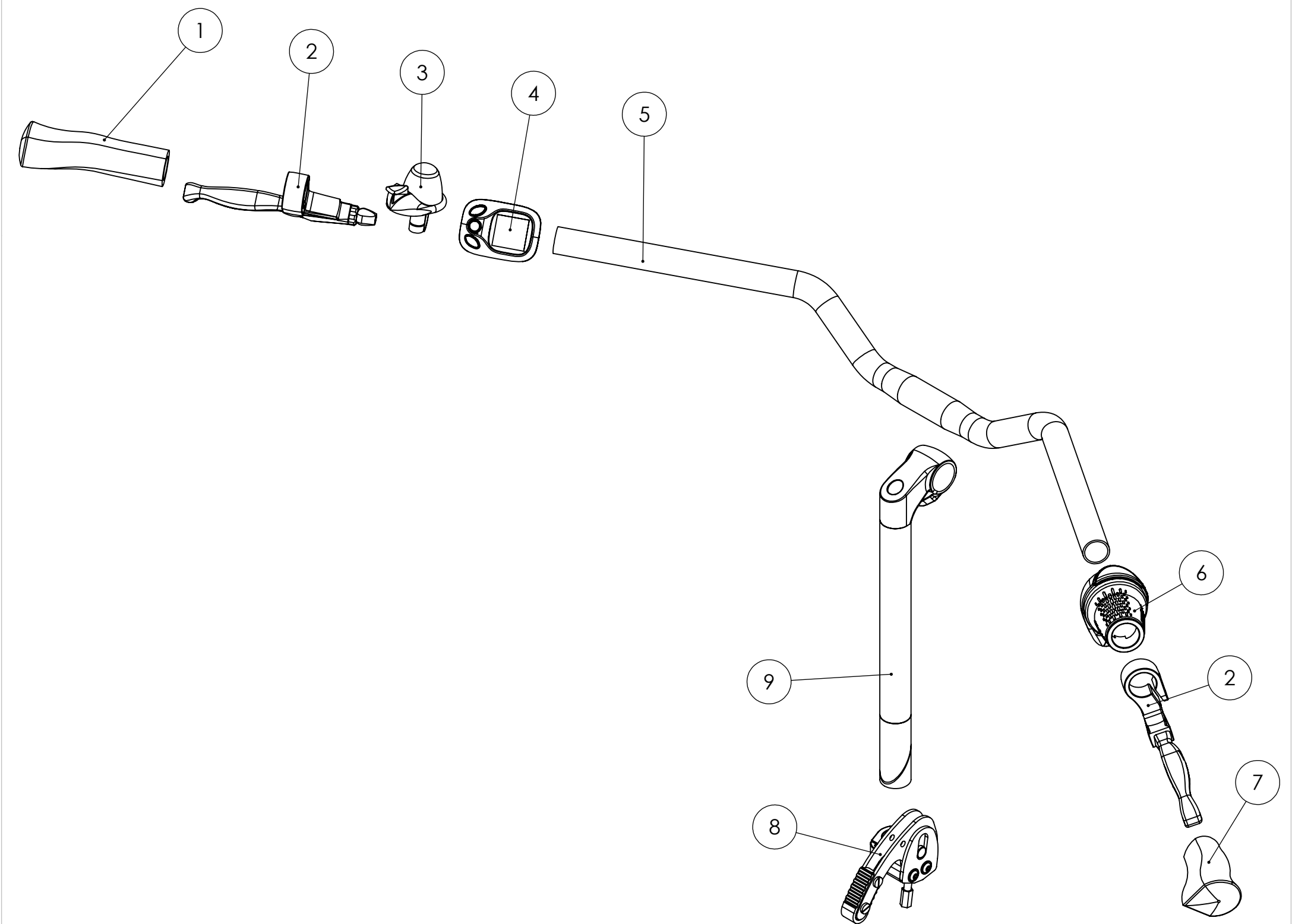


9	426809	Stem Ø25,4
8	406153VM	Parking brake
7	433013	Grip right
6	431223	Gripshift Shimano 8V
5	401424	Handlebar Ø22.2
4	401927	Display Bafang DP C15
3	426827	Bell decibel
2	401929	Mechanical brake lever Bafang set
1	433012	Grip left
POS	ART. NO.	NAME

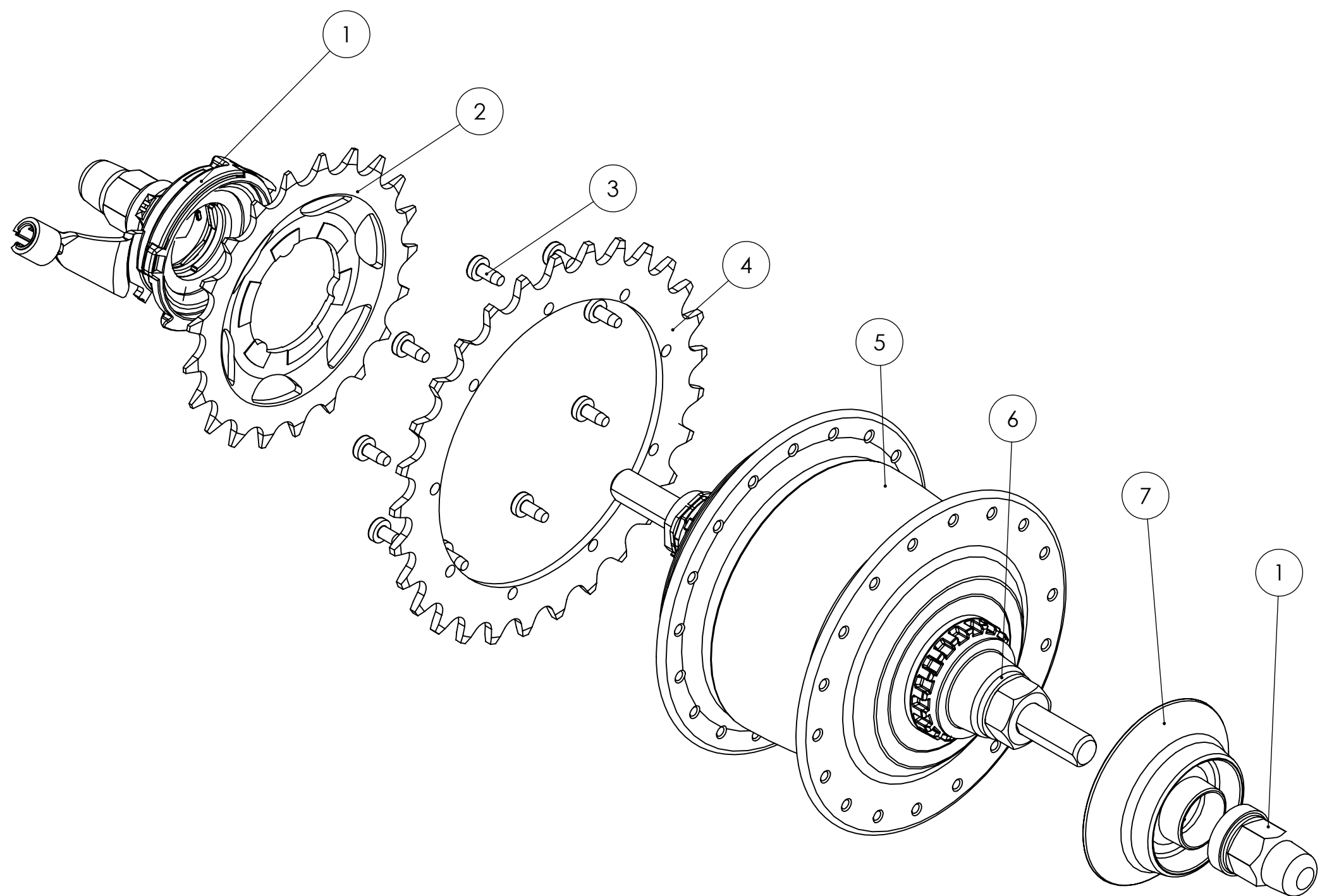


7	496024	Cover Nexus 8V
6	410080	Plain washer DIN125-1A zinc plated M10
5	431209	Gear hub Nexus 8V freewheel
4	497052	Sprocket 1/8" 29T Nexus 8
3	410163	Raised cheese head thread rolling screw DIN 7500-1 zinc plated M3.5 X 8 8.8
2	401689	Sprocket 3/32" 21T spline
1	409805	Shifter-unit Shimano Nexus 8
POS	ART. NO.	NAME

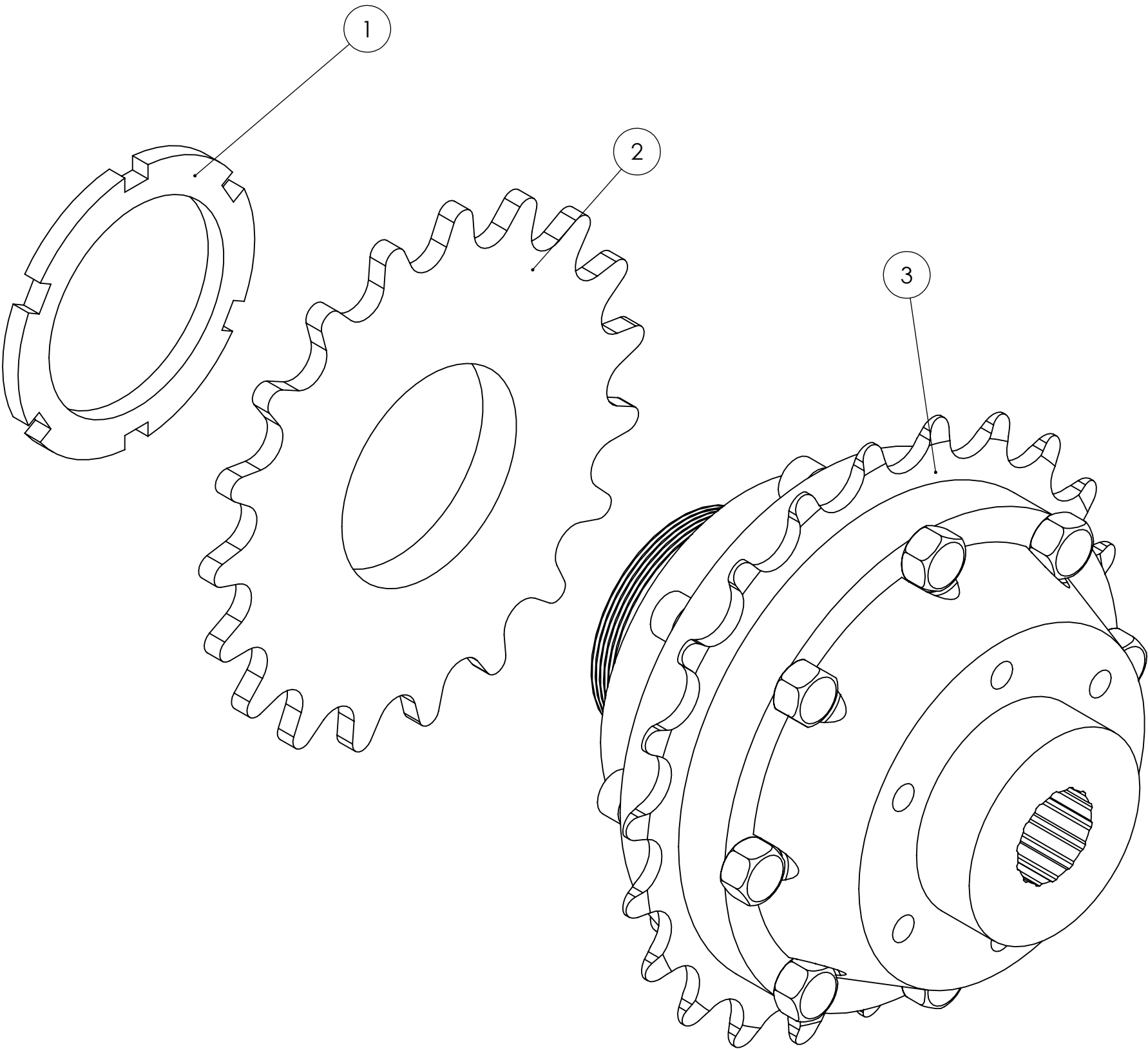


3	406405	Differential with spline
2	480797	Sprocket 3/32 19T threaded
1	480796	Lock nut bsa 1 3/8 X 24
POS	ART. NO.	NAME

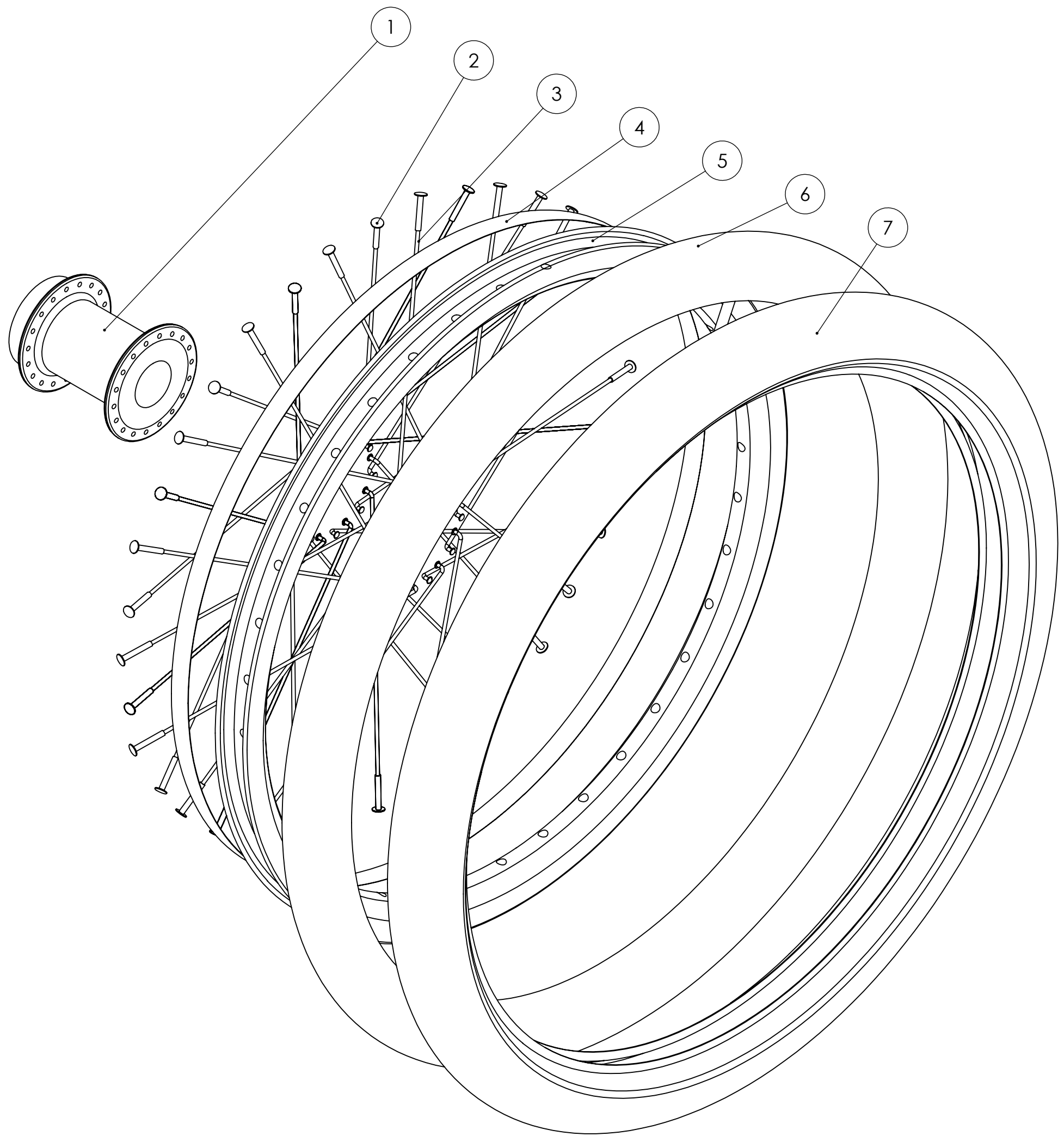


7	401339	Tire 20" Schwalbe Marathon Plus 47-406
6	401346	Inner tube 20" Schwalbe DV 55-406
5	406227	Rim 20" X-plorer
4	426530	Rim tape 15mm
3	426627	Spoke 13x178 stainless steel
2	435067	Nipple 13G
1	406400	Hub rear
POS	ART. NO.	NAME

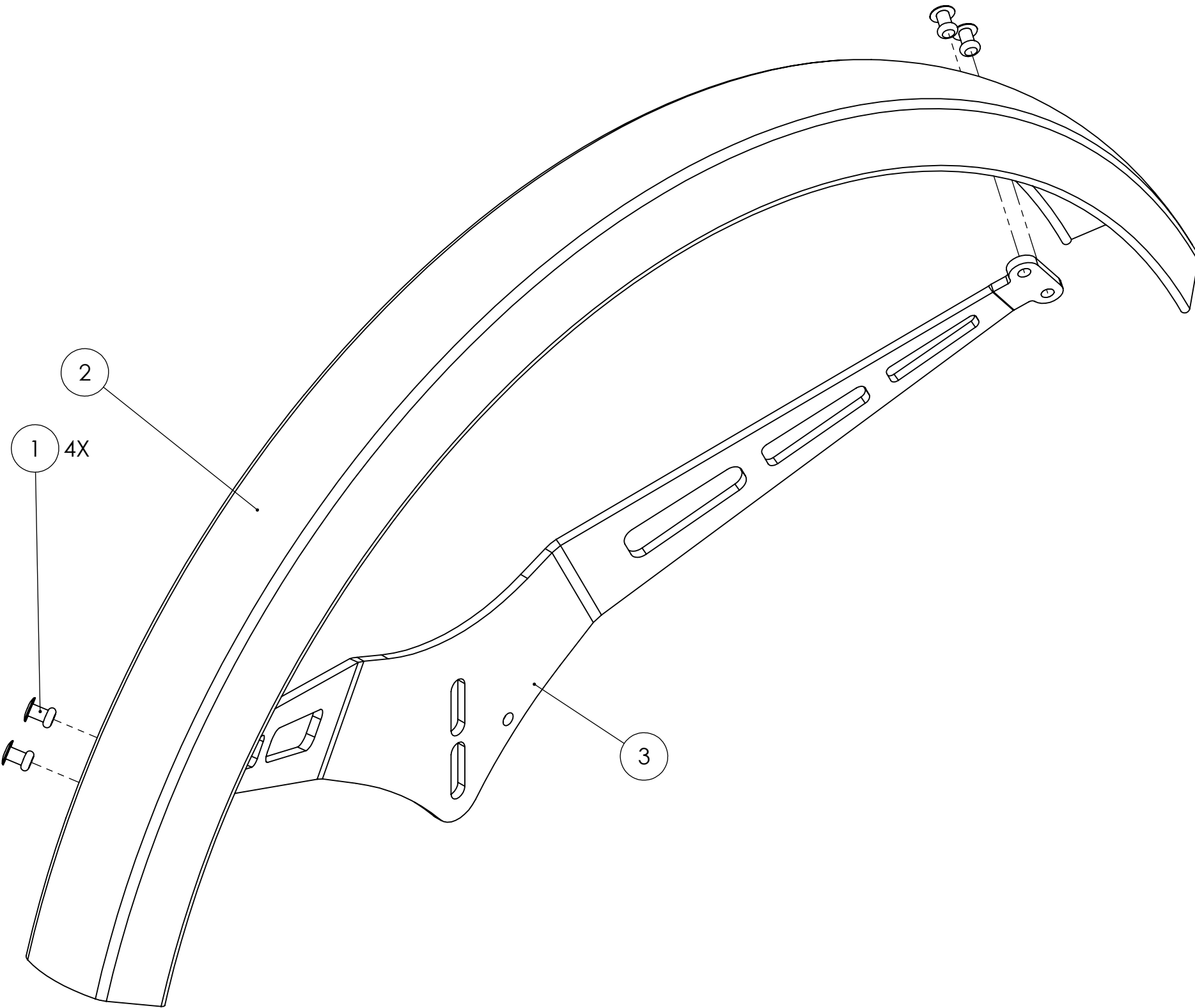

3	406208	Fender bracket back
2	406234	Fender 20" back
1	427338	Blind rivet aluminium Ø4 X 10
POS	ART. NO.	NAME

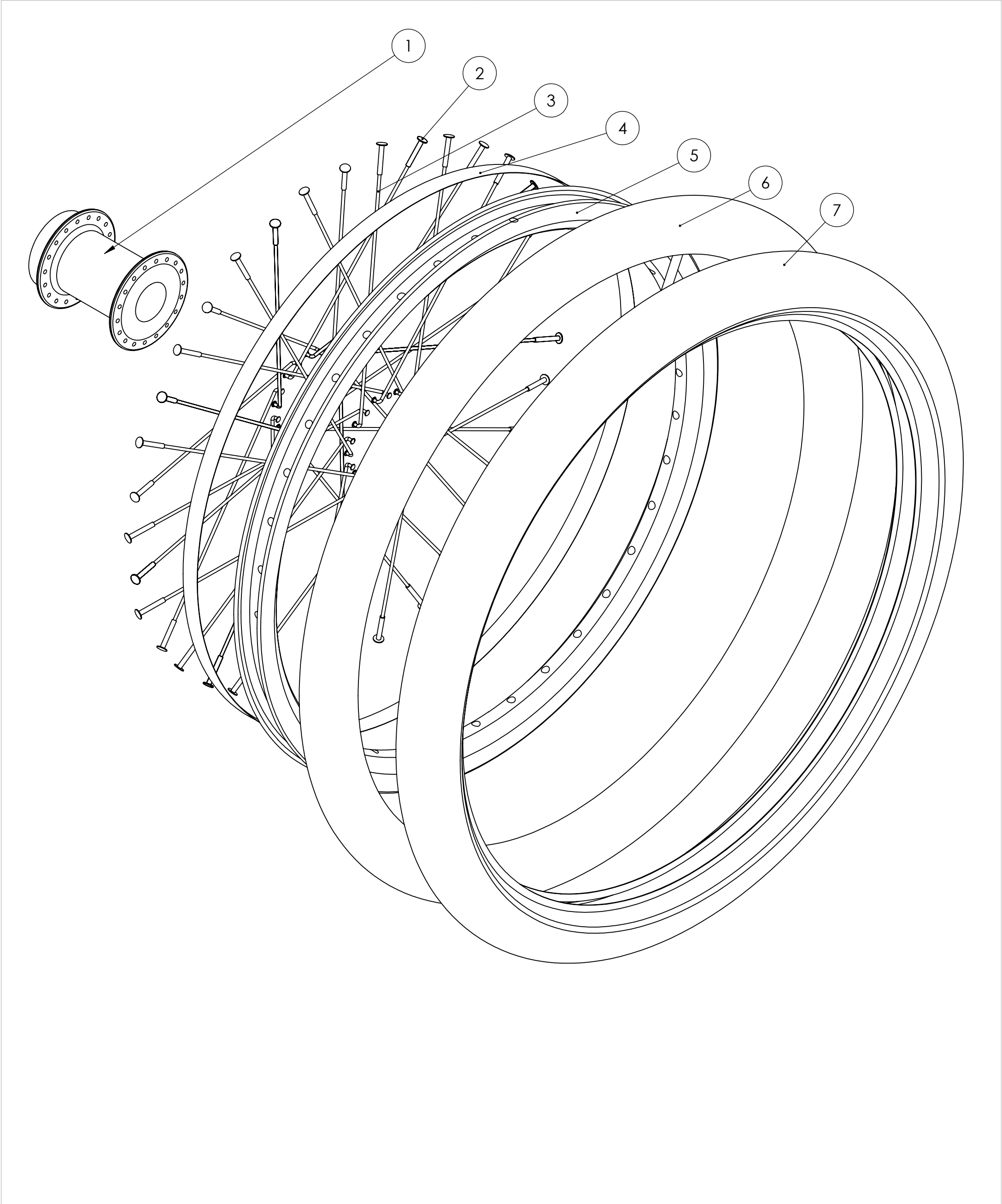

7	401339	Tire 20" Schwalbe Marathon Plus 47-406
6	401346	Inner tube 20" Schwalbe DV 55-406
5	406227	Rim 20" X-plorer
4	426530	Rim tape 15mm
3	426627	Spoke 13x178 stainless steel
2	435067	Nipple 13G
1	406400	Hub rear
POS	ART. NO.	NAME

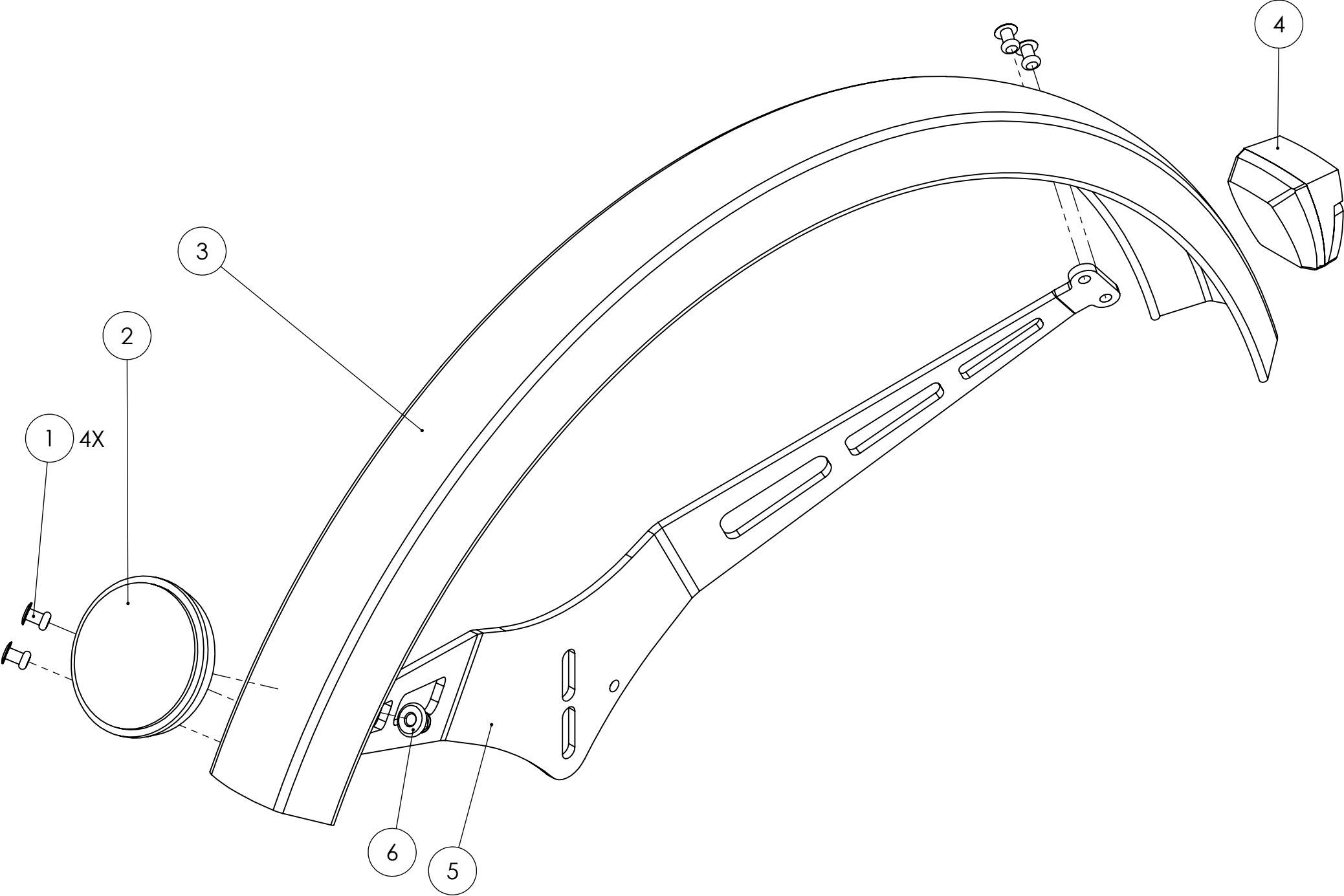

6	410212	Prevailing torque hex nut with flange DIN 6926 zinc plated M5 8.8
5	406208	Fender bracket back
4	401670	Rear light E-bike
3	406234	Fender 20" back
2	406293	Reflector white round 55mm
1	427338	Blind rivet aluminium Ø4 X 10
POS	ART. NO.	NAME