



Exploded views

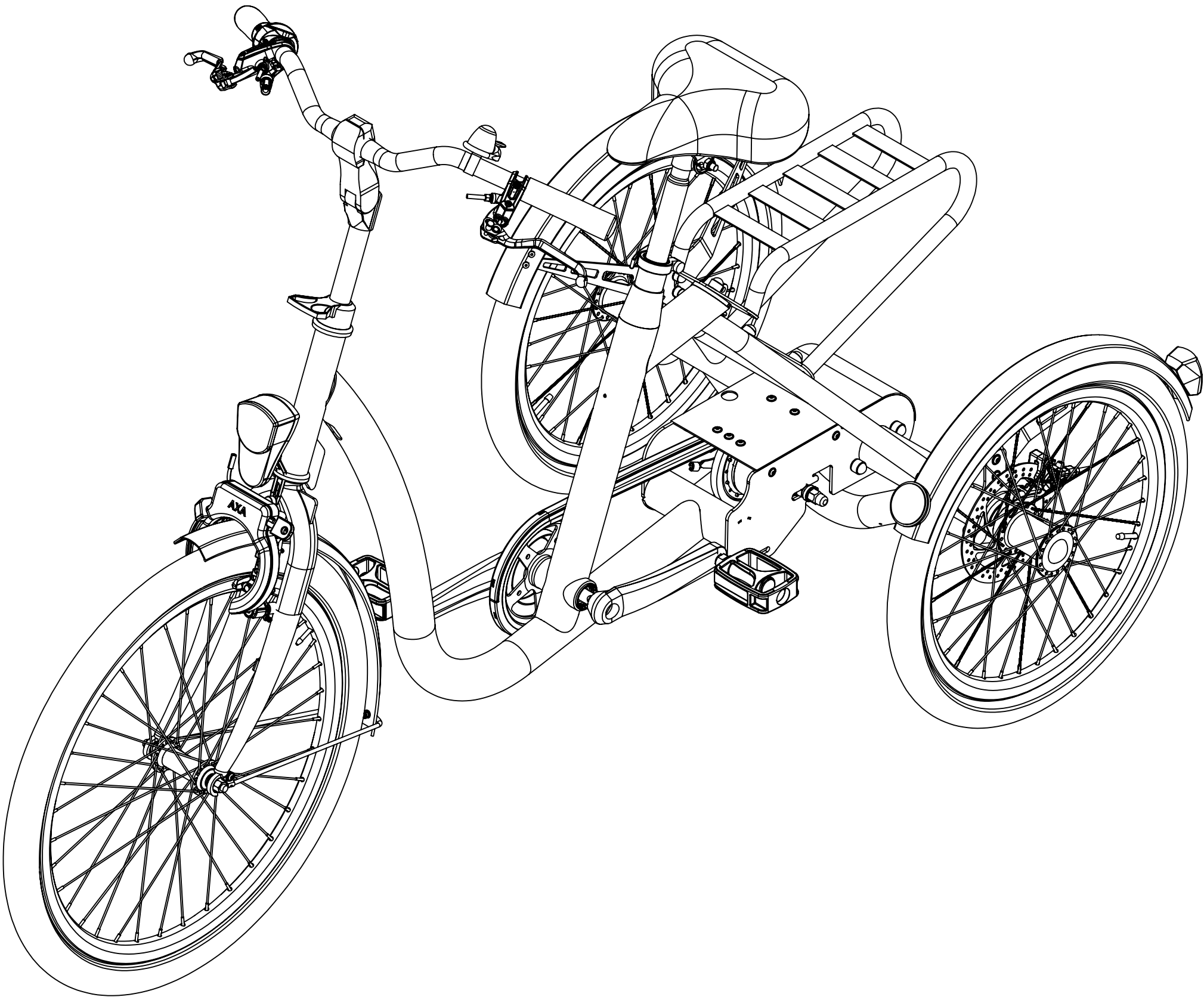


Data of issue: 25-6-2024	Be advised that in some cases the exploded views, might contain other parts beside what is shown in the illustration due to interchangeable accessories and options	 Huka BV Munsterstraat 13 7575 ED Oldenzaal The Netherlands
Model: City	This publication has been designed for use by Huka dealers, and may be updated periodically. Huka policy is one of continuous product improvement and the right is reserved to alter specifications at any time without prior notice. Whilst every care has been taken to ensure correctness of information, it is impossible to guarantee complete freedom from omissions and errors, or to accept liability arising from such errors and omissions	
Production period: 2020-present day		
Huka BV, Münsterstraat 13, 7575 ED Oldenzaal, tel: +31 (0)541-572472, info@huka.nl		

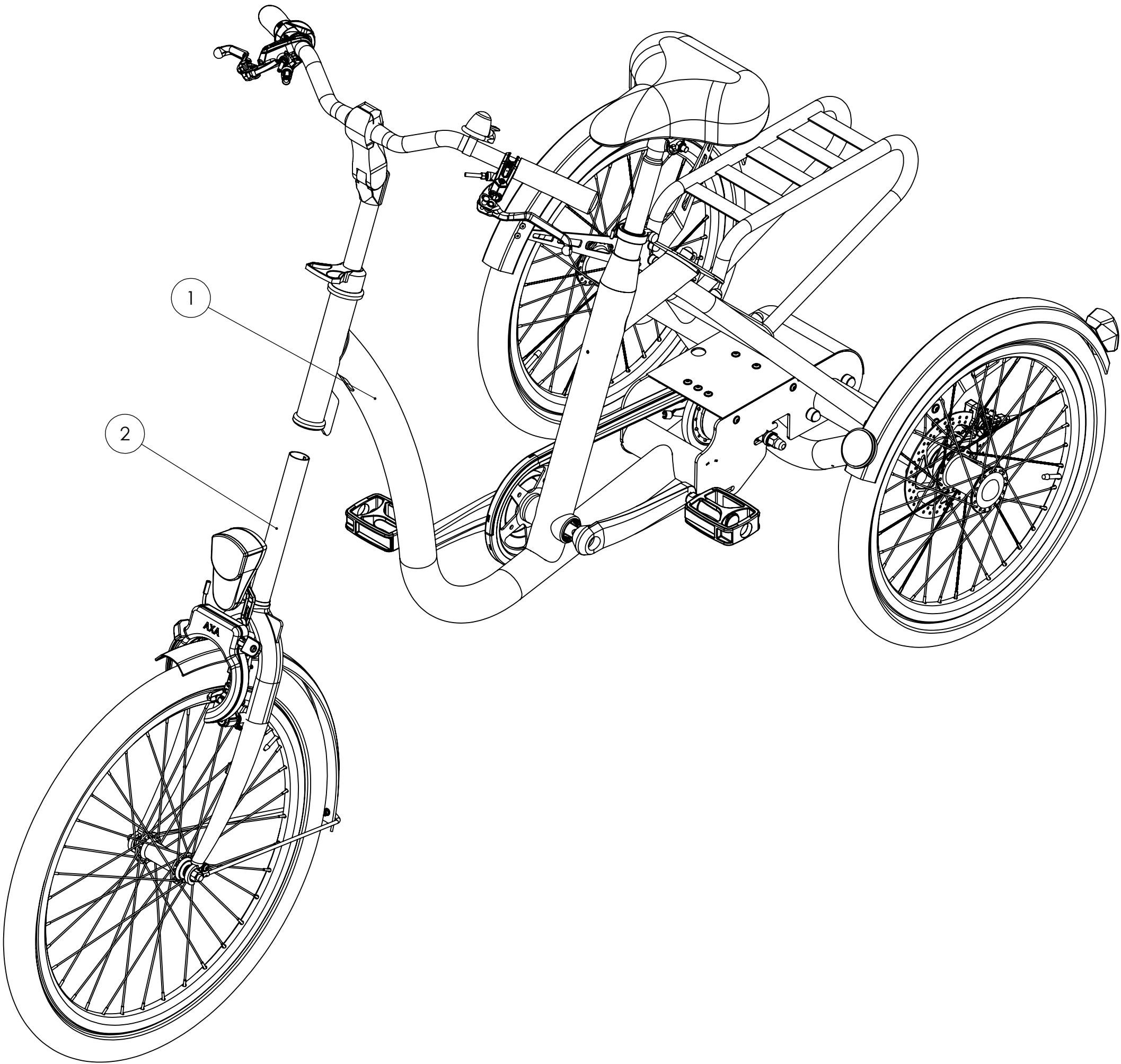


2	-	Front fork City
1	-	Frame City
POS	ART. NO.	NAME


Huka BV, Münsterstraat 13, 7575 ED Oldenzaal, tel: +31 (0)541-572472, info@huka.nl

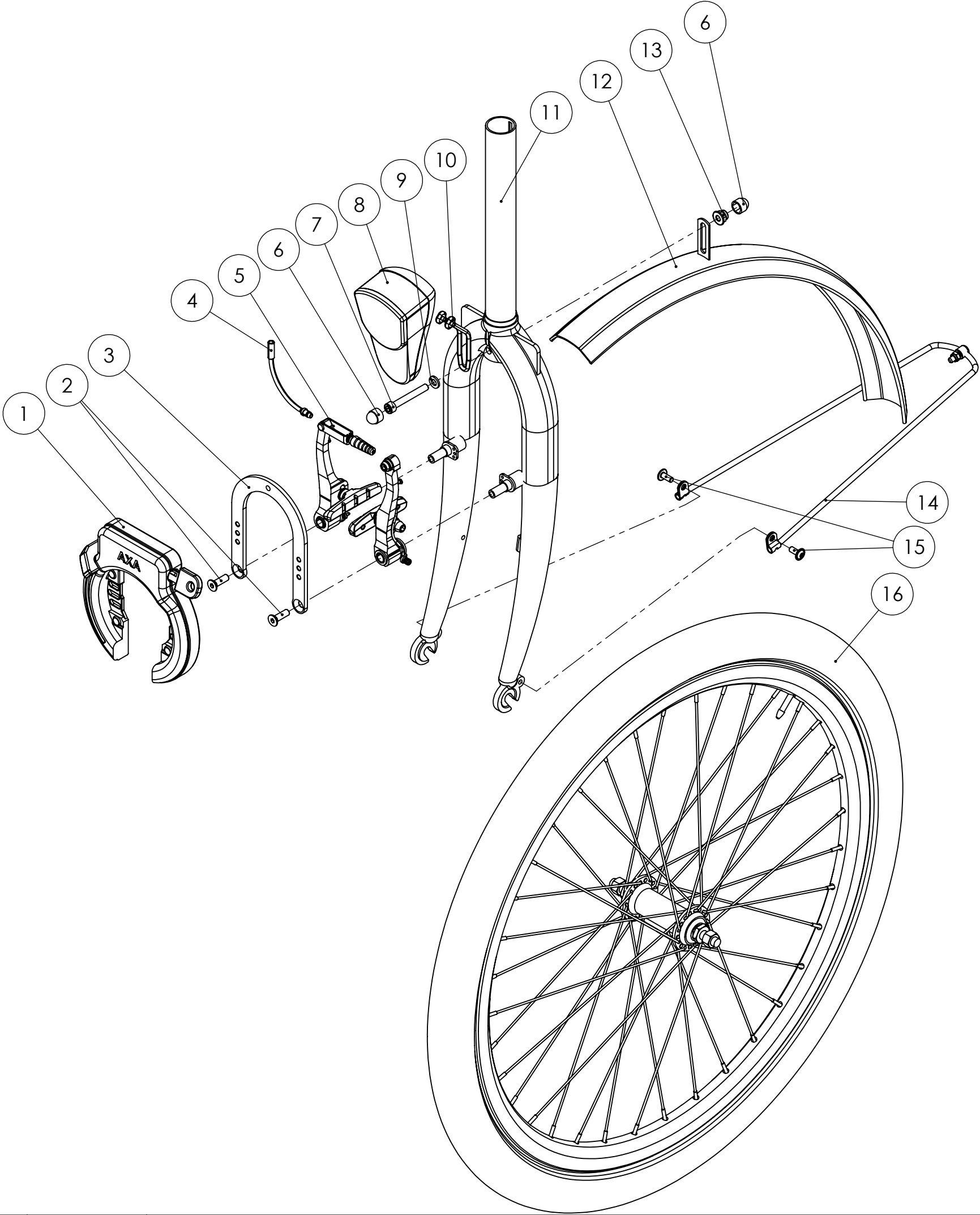

16	411214	Wheel 24" front
15	438268	Hex socket button head screw with flange ISO 7380-2 zinc plated M5 X 12 8.8
14	426763	Fender bracket
13	438267	Prevailing torque hex nut with flange DIN 6926 zinc plated M6 8.8
12	406231	Fender 24" front
11	404367	Front fork 24"
10	401624	Front light bracket
9	410109	Plain washer DIN125-1A zinc plated M6
8	432976	Front light battery
7	412963	Hex socket cap screw DIN 912 zinc plated M6 X 45 8.8
6	475222	Bolt protection cap M6
5	401528	Mechanical v-brake Shimano BR-T4000
4	435063	V-brake noodle 90°
3	404427	Ring lock bracket
2	464058	Hex socket countersunk head screw ISO 10642 zinc plated M6 X 20 8.8
1	426582	Ringlock AXA Solid
POS	ART. NO.	NAME

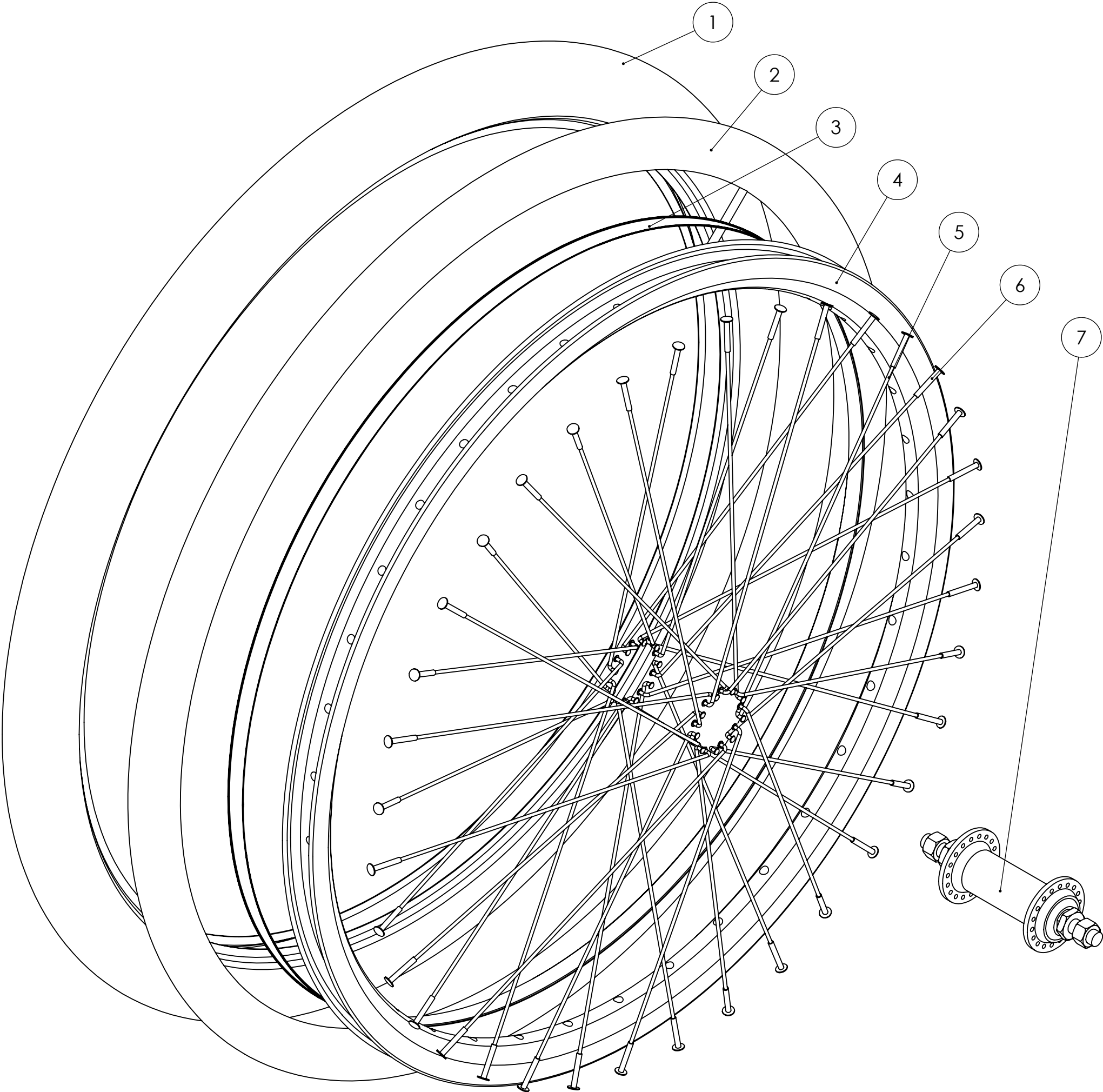


7	401702	Hub front Shimano TX500
6	435067	Nipple 13G
5	401762	Spoke 13x230 stainless steel
4	426506	Rim 24" X-plorer
3	426530	Rim tape 15mm
2	401350	Inner tube 24" Schwalbe DV 55-507
1	401340	Tire 24" Schwalbe Marathon Plus 47-507
POS	ART. NO.	NAME

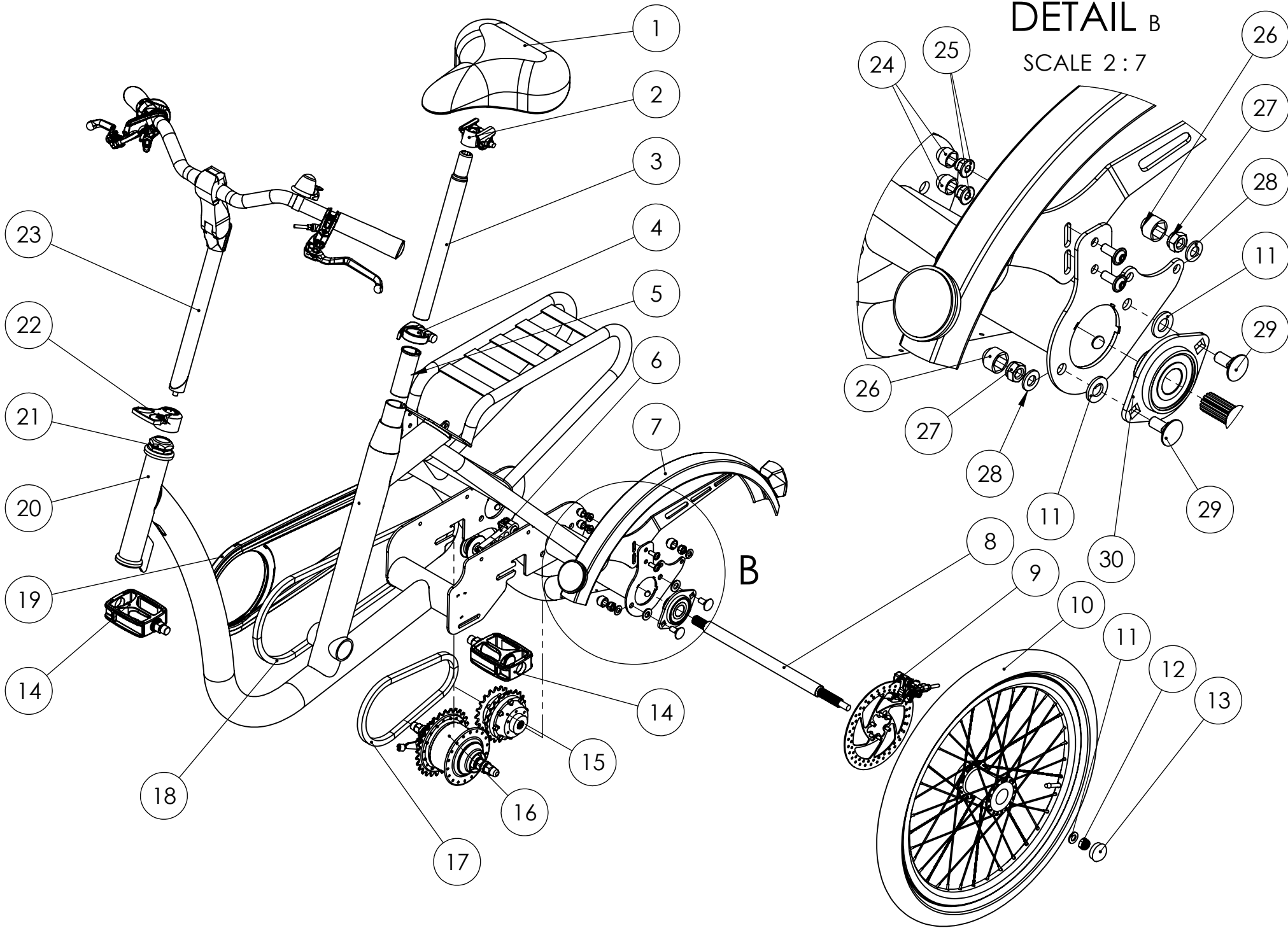


30	406217	Bearing flanged
29	410227	Carriage bolt DIN 603 zinc plated M8 X 20 4.8
28	410110	Plain washer DIN125-1A zinc plated M8
27	410204	Prevailing torque hex nut DIN 985 zinc plated M8
26	464065	Bolt protection cap M8
25	438267	Prevailing torque hex nut with flange DIN 6926 zinc plated M6 8.8
24	475222	Bolt protection cap M6
23	-	Steer City
22	401666	Cable guide head tube 1 1/8"
21	409302	Headset bearing threaded 1 1/8 inch
20	401652B	Welded frame
19	401665	Chainguard chain glider
18	408058	Chain 1/2 x 3/32
17	426808	Chain 1/2 x 1/8
16	406120VM	Gear hub Nexus 8V freewheel set
15	406405VM	Differential with spline
14	401645	Pedal set standard
13	411300	Protection cap Ø30
12	412970	Prevailing torque hex nut DIN 985 zinc plated M10
11	410080	Plain washer DIN125-1A zinc plated M10
10	401662	Wheel 20" Rear
9	406266	Hydraulic disk brake set Bengal rear right
8	406410	Rear axle left
7	406330	Fender left back
6	406249	Chain tensioner
5	451409	Seatpost spacer Ø27,2 - Ø30
4	401602	Seatpost clamp
3	408114	Seatpost rigid Ø27.2x300
2	411058	Saddle mounting set
1	401637	Saddle Velo Plush
POS	ART. NO.	NAME

DETAIL D


SCALE 1 : 6

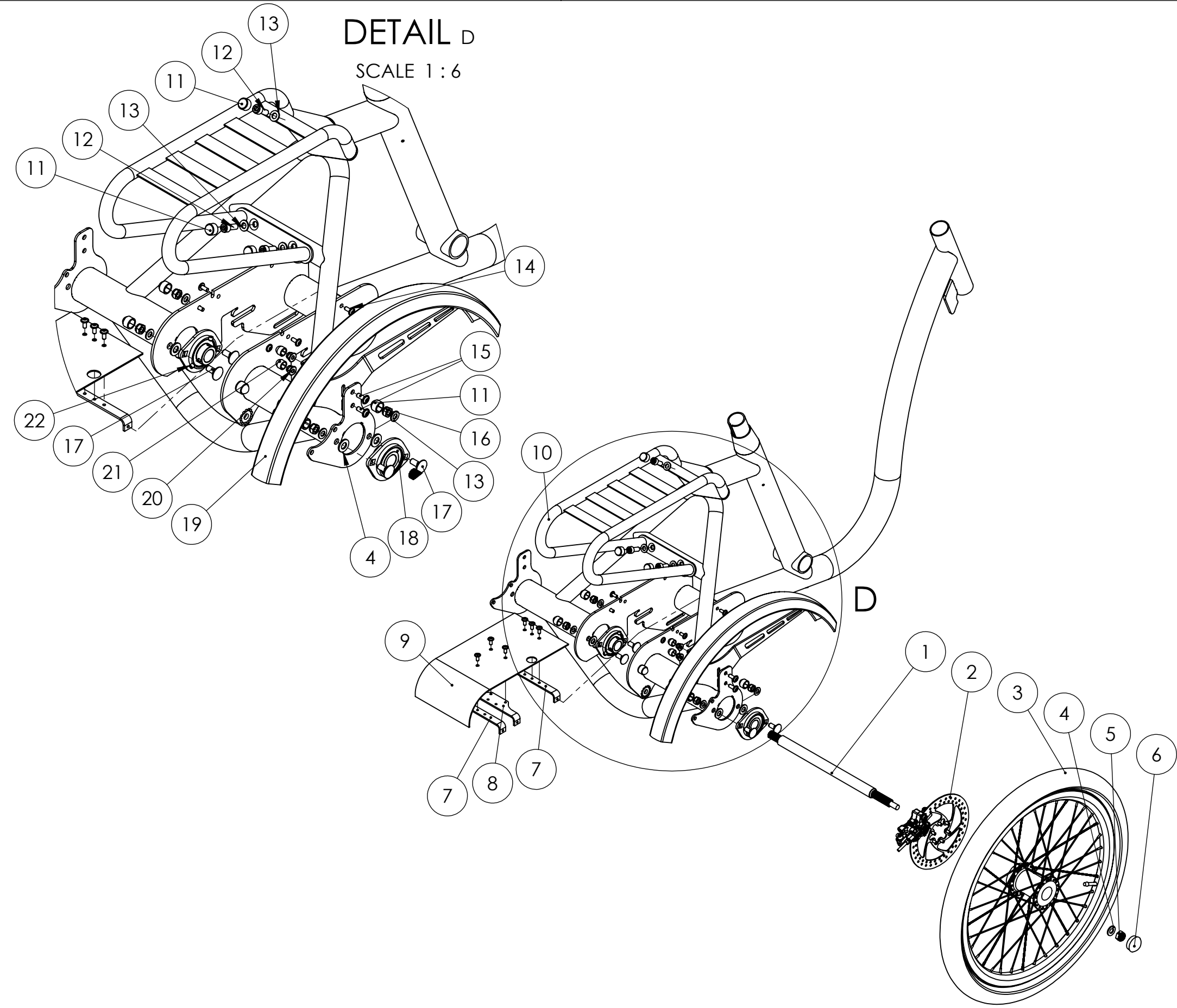


22	412965	Hex socket set screw DIN 913 zinc plated M6 X 6
21	475222	Bolt protection cap M6
20	438267	Prevailing torque hex nut with flange DIN 6926 zinc plated M6 8.8
19	406331	Fender right back
18	406217	Bearing flanged
17	410227	Carriage bolt DIN 603 zinc plated M8 X 20 4.8
16	410204	Prevailing torque hex nut DIN 985 zinc plated M8
15	451345	Hex socket button head screw with flange ISO 7380-2 zinc plated M6 X 16 8.8
14	438268	Hex socket button head screw with flange ISO 7380-2 zinc plated M5 X 12 8.8
13	410110	Plain washer DIN125-1A zinc plated M8
12	410085	Hex socket cap screw DIN 912 zinc plated M8 X 20 8.8
11	464065	Bolt protection cap M8
10	401653B	Lasassy bagagedrager C3S
9	401656B	Chainguard back
8	401658	Support strip brake splitter
7	401655	Support strip chainguard
6	411300	Protection cap Ø30
5	412970	Prevailing torque hex nut DIN 985 zinc plated M10
4	410080	Plain washer DIN125-1A zinc plated M10
3	401663	Wheel 20" Rear
2	406266	Hydraulic disk brake set Bengal rear right
1	406411	Rear axle right
POS	ART. NO.	NAME

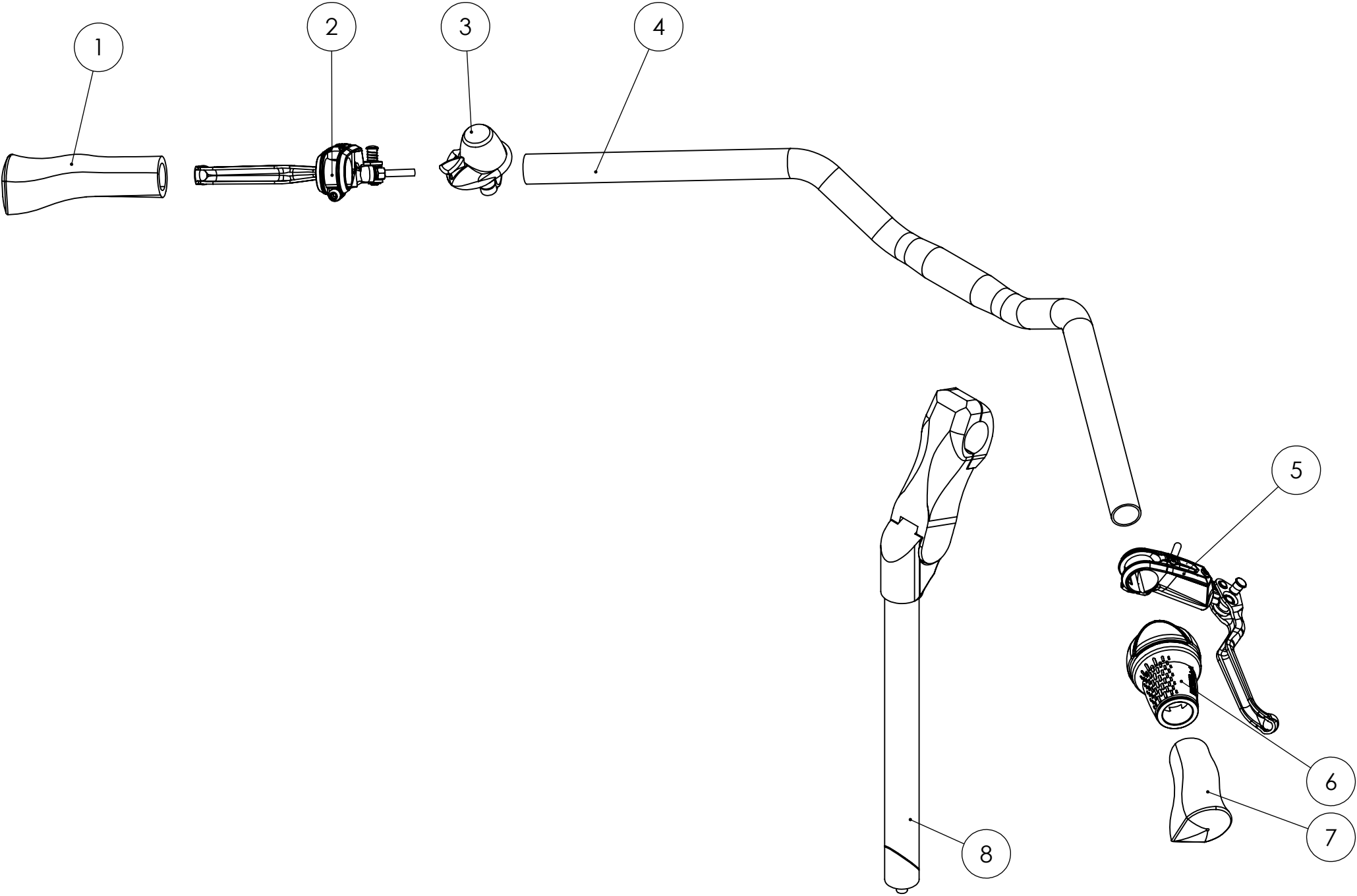


8	409039	Stem ergonomic Ø25,4 adjustable
7	433013	Grip right
6	431223	Gripshift Shimano 8V
5	406266	Hydraulic disk brake set Bengal rear right
4	401424	Handlebar Ø22.2
3	426827	Bell decibel
2	406254	Mechanical brake lever left
1	433012	Grip left
POS	ART. NO.	NAME

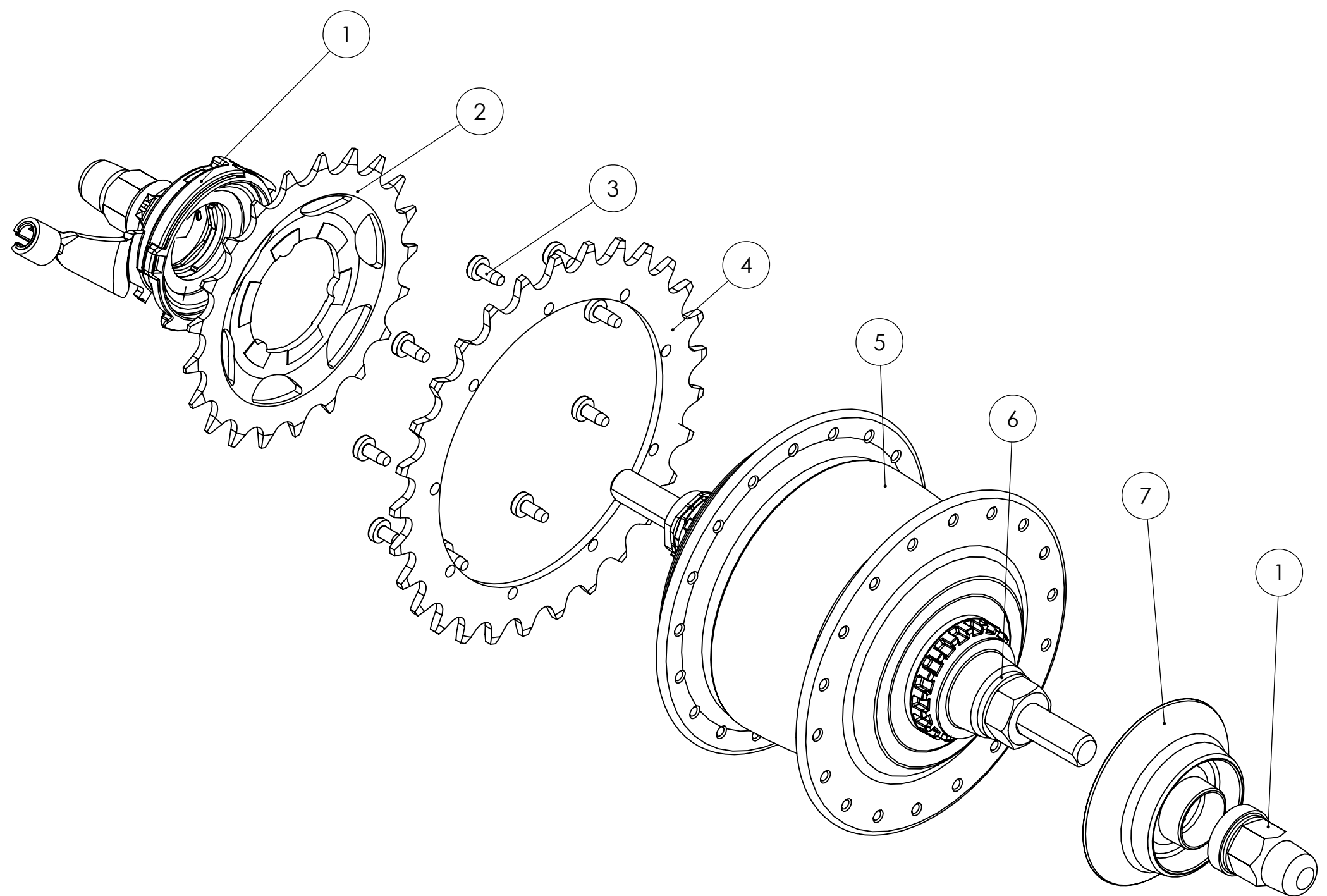


7	496024	Cover Nexus 8V
6	410080	Plain washer DIN125-1A zinc plated M10
5	431209	Gear hub Nexus 8V freewheel
4	497052	Sprocket 1/8" 29T Nexus 8
3	410163	Raised cheese head thread rolling screw DIN 7500-1 zinc plated M3.5 X 8 8.8
2	401689	Sprocket 3/32" 21T spline
1	409805	Shifter-unit Shimano Nexus 8
POS	ART. NO.	NAME

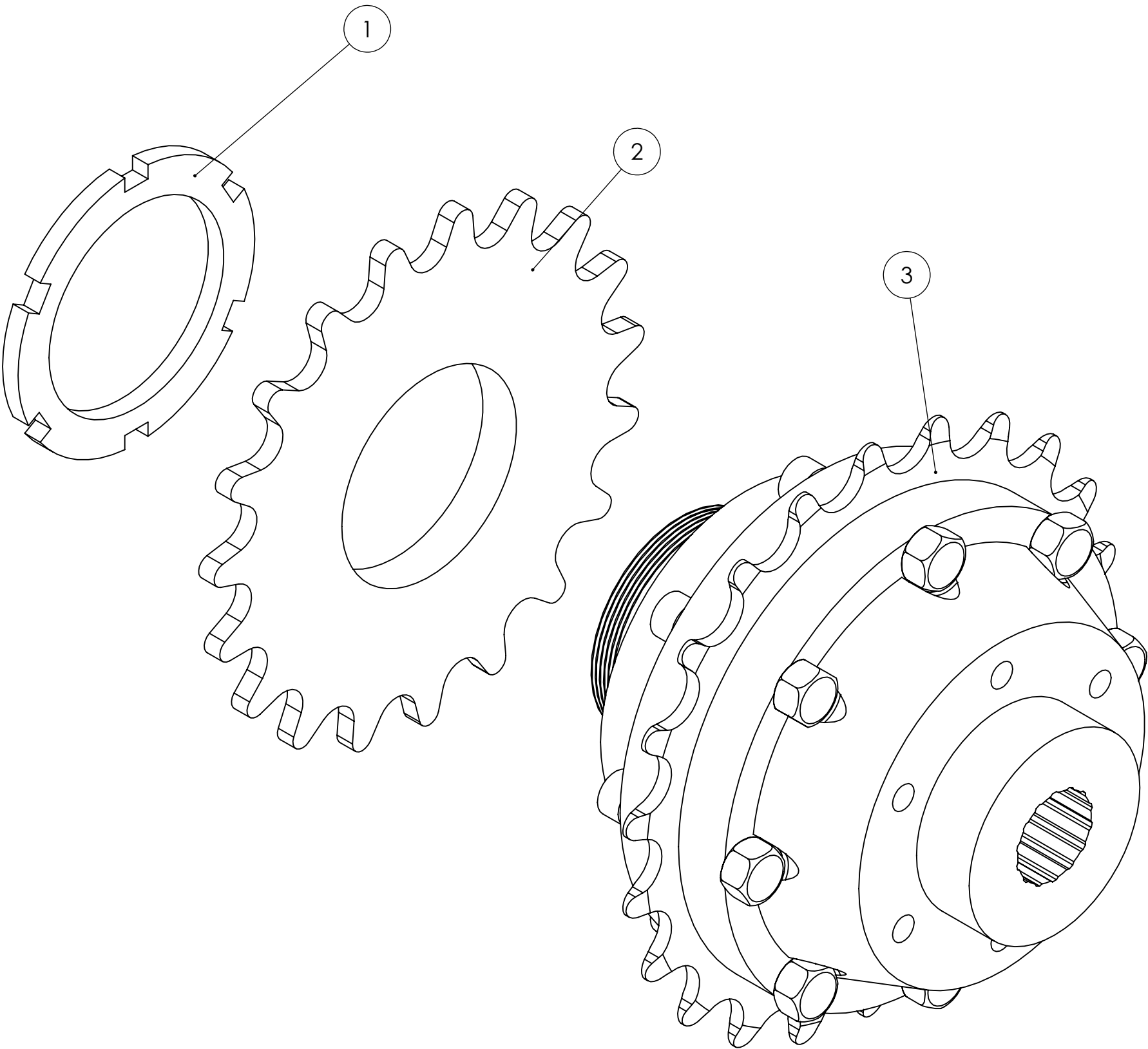


3	406405	Differential with spline
2	480797	Sprocket 3/32 19T threaded
1	480796	Lock nut bsa 1 3/8 X 24
POS	ART. NO.	NAME

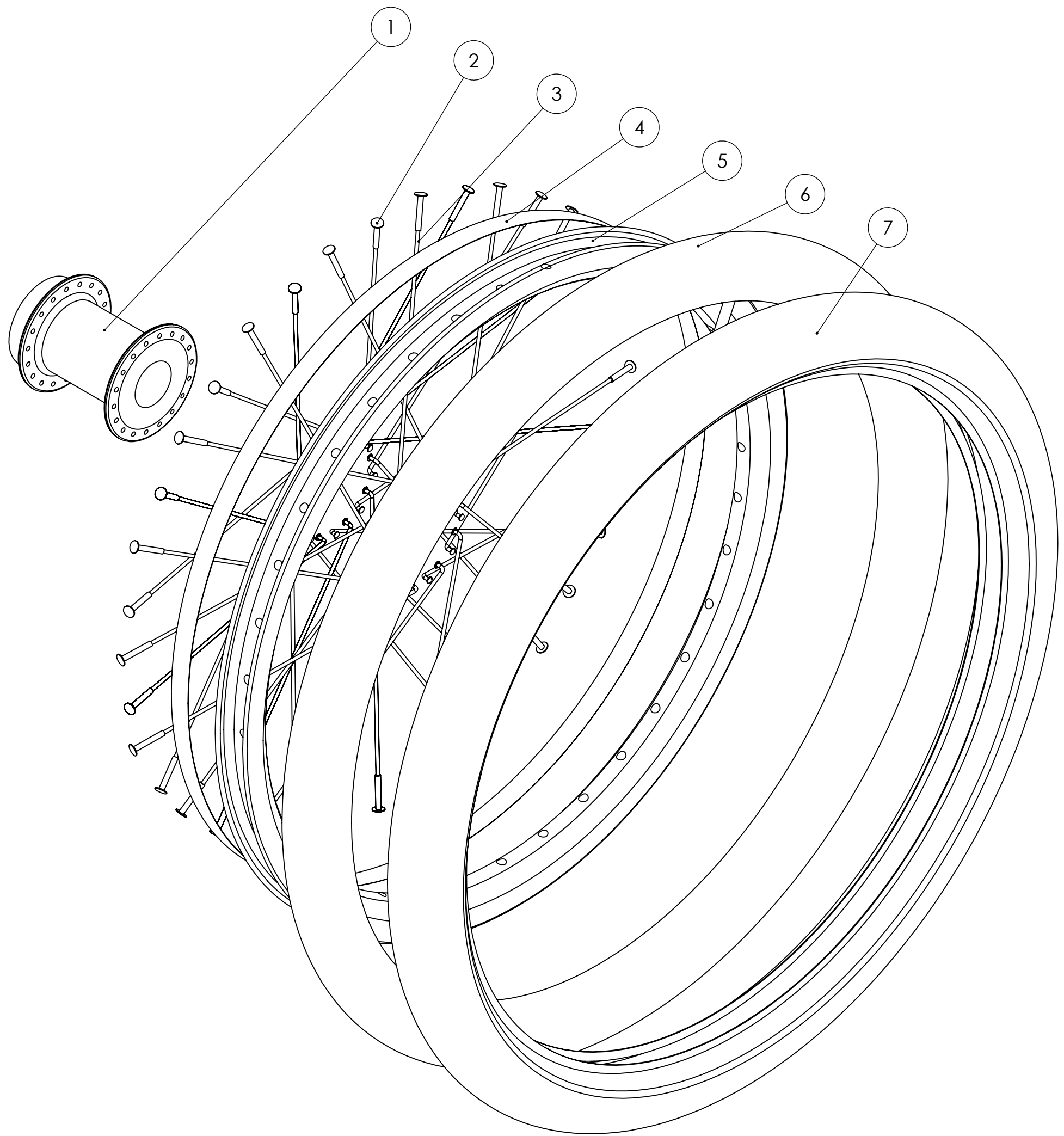


7	401339	Tire 20" Schwalbe Marathon Plus 47-406
6	401346	Inner tube 20" Schwalbe DV 55-406
5	406227	Rim 20" X-plorer
4	426530	Rim tape 15mm
3	426627	Spoke 13x178 stainless steel
2	435067	Nipple 13G
1	406400	Hub rear
POS	ART. NO.	NAME

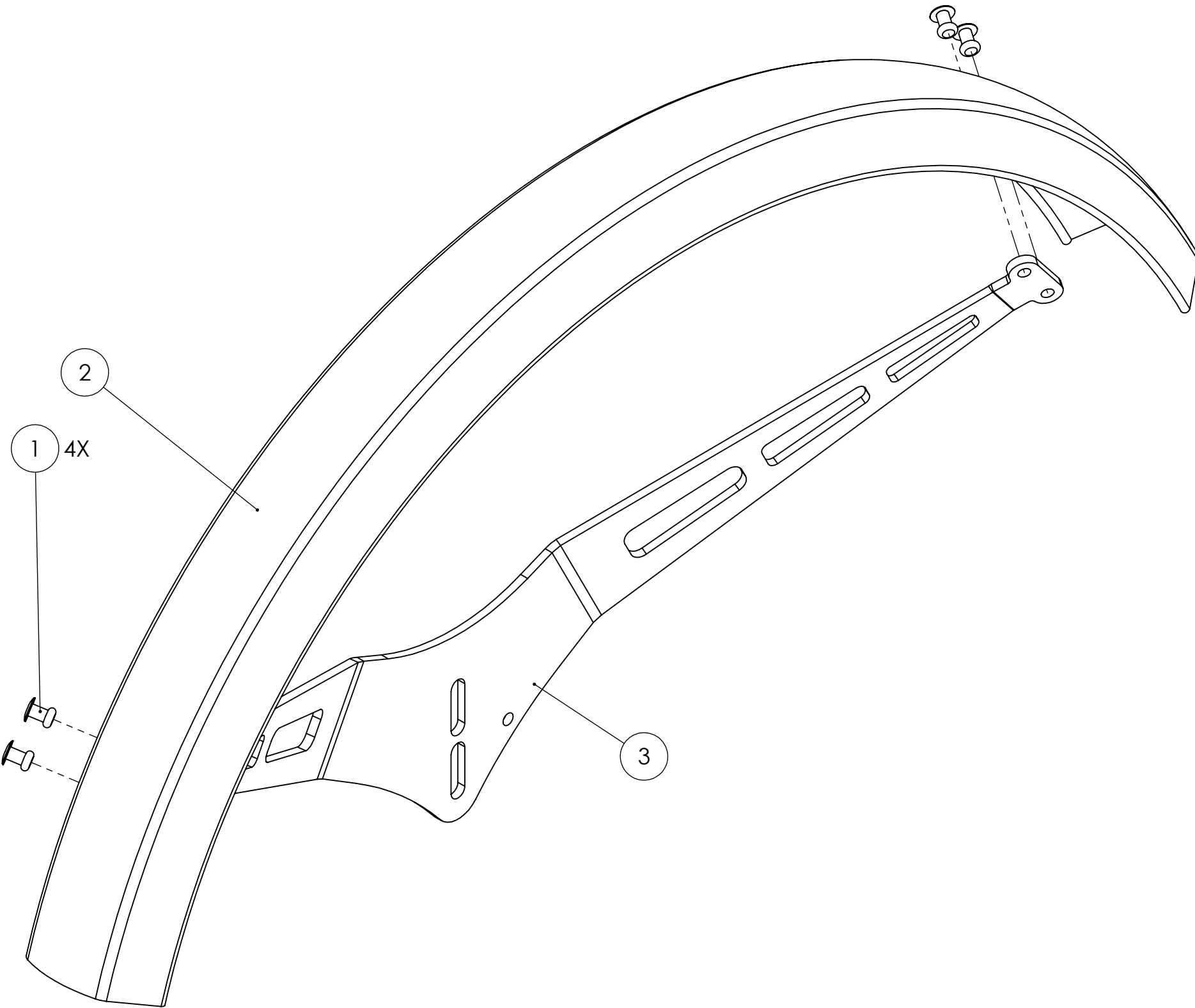

3	406208	Fender bracket back
2	406234	Fender 20" back
1	427338	Blind rivet aluminium Ø4 X 10
POS	ART. NO.	NAME

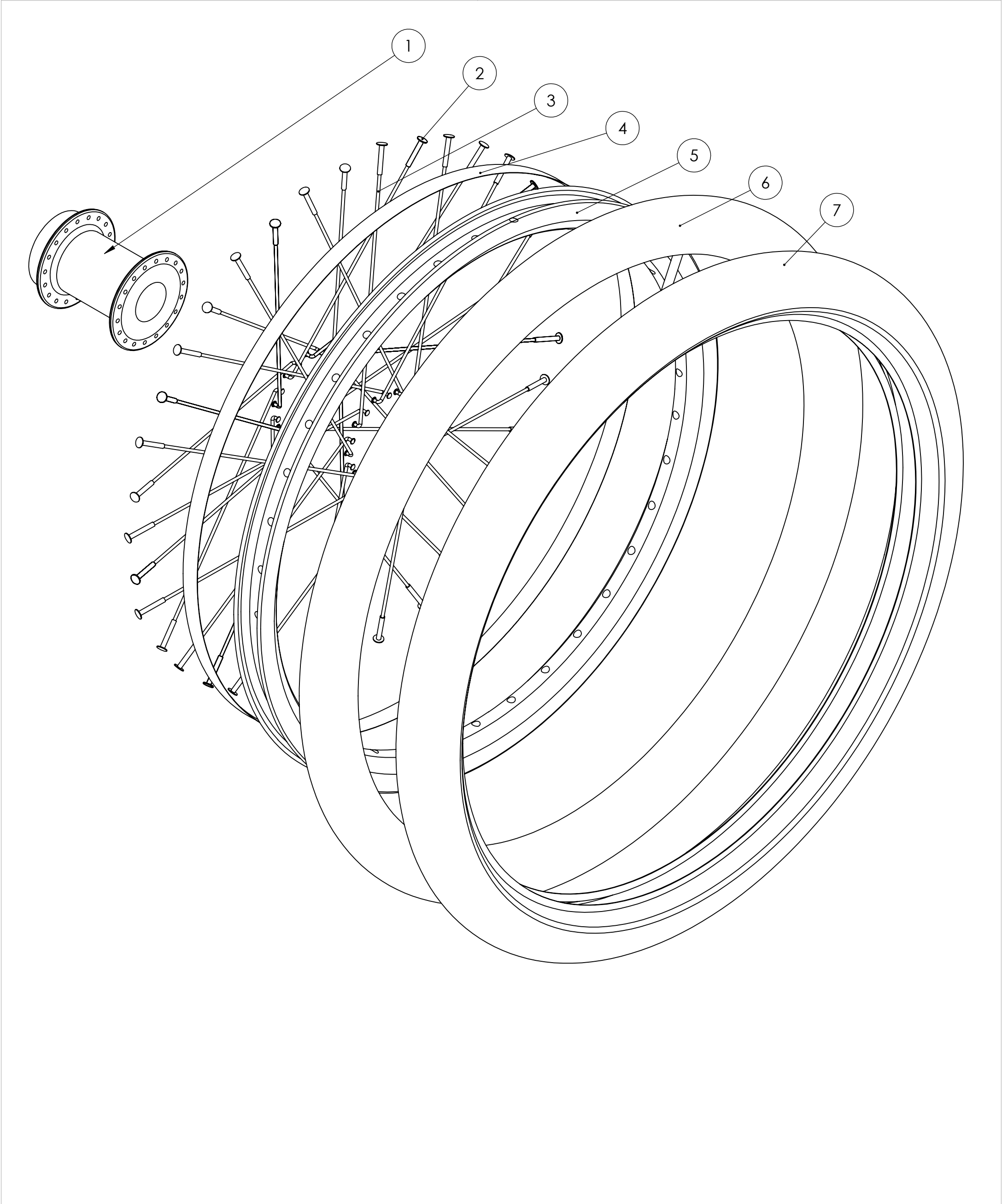

7	401339	Tire 20" Schwalbe Marathon Plus 47-406
6	401346	Inner tube 20" Schwalbe DV 55-406
5	406227	Rim 20" X-plorer
4	426530	Rim tape 15mm
3	426627	Spoke 13x178 stainless steel
2	435067	Nipple 13G
1	406400	Hub rear
POS	ART. NO.	NAME

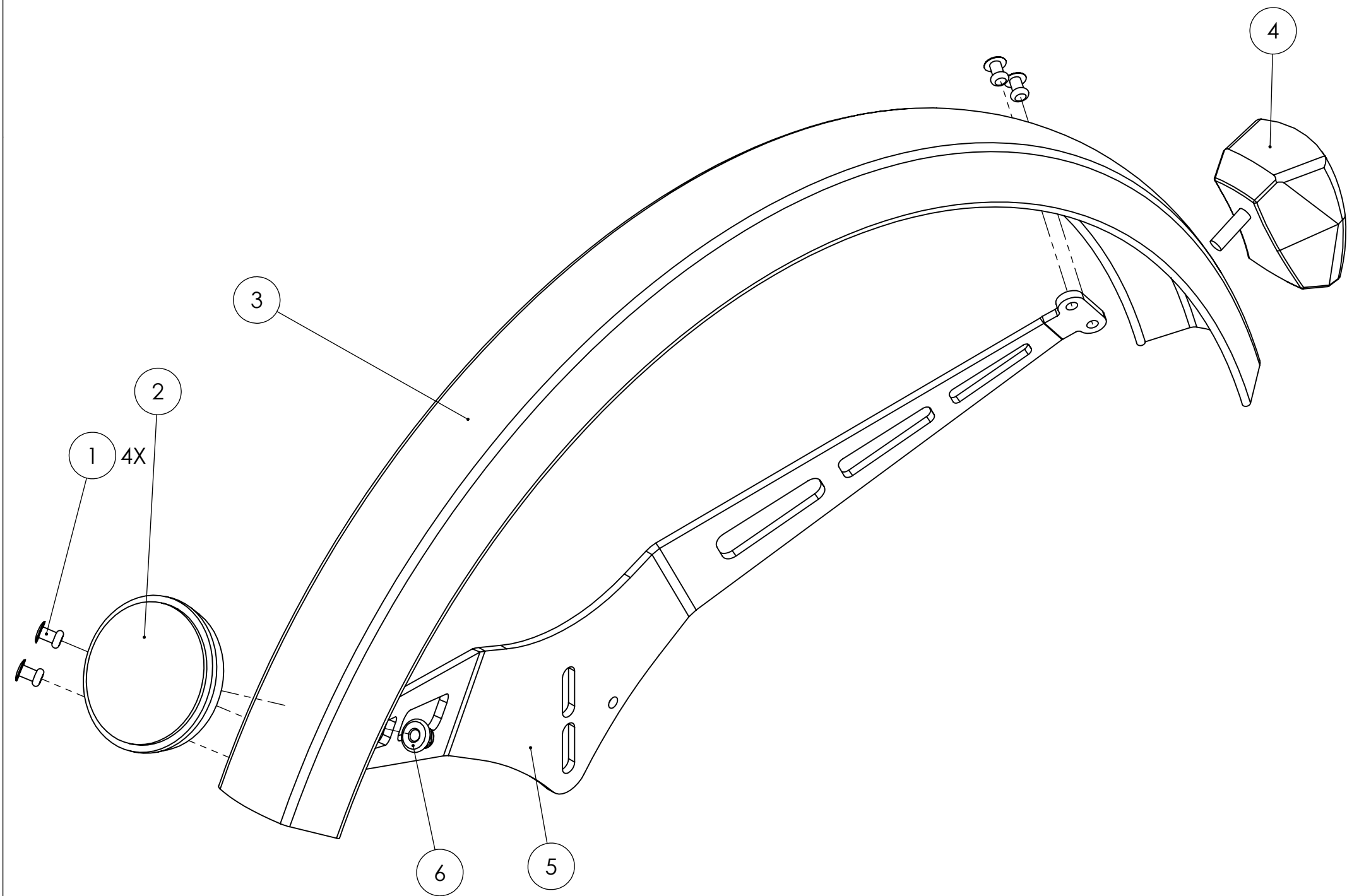

6	410212	Prevailing torque hex nut with flange DIN 6926 zinc plated M5 8.8
5	406208	Spatborddrager achter zwart
4	435143	Rear light battery
3	406234	Fender 20" back
2	406293	Reflector white round 55mm
1	427338	Blind rivet aluminium Ø4 X 10
POS	ART. NO.	NAME